



# Nue Fusion 600 Microcurrent Face & Body System



© Copyright 2014 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

ver2015-1

# INTRODUCTION

## *OUR PROMISE TO YOU!*

We are dedicated to providing reliable, affordable, and results-oriented professional equipment. Since 2003, we have been providing superior equipment, such as: microdermabrasion, microcurrent, oxygen infusion, LED Light Therapy along with performance professional skin care.

Our parent company Lifeline Medical, Inc. has been in business since 1991. A strong commitment to customer service has contributed to our success. I, as owner and President, pledge to continue this tradition and to make Skin for Life the premier customer service oriented company in this industry.

Our dedicated employees understand the importance of customer focus and we treat all our customers with integrity and respect. We realize that our ability to grow as a company is directly related to customer satisfaction and that you make the final choice when selecting a vendor.

We look forward to the opportunity to support your business with quality equipment, supplies, and skin care. Thank you for giving us a chance to earn your trust.



Tushar Shah  
President and Founder of Skin for Life

Please visit us online at:  
[www.skinforlife.com](http://www.skinforlife.com) and [www.skinforlifepurescience.com](http://www.skinforlifepurescience.com)

# NUE FUSION 600 MICROCURRENT

Introduction .....	2
Important Safety Instructions.....	4
What is Microcurrent?.....	5
Features & Benefits of the Nue Fusion 200.....	6
Contraindications & Precautions.....	6
Functions - TEST MODE .....	7
Functions - LCD panel.....	8
Microcurrent Procedure & Techniques.....	9-10
Facial Muscles.....	11
Microcurrent Suggested Facial Movements - Ball Wands.....	12-15
Skin Tightening & Firming with Straight Wands.....	16-19
Facial Maintenance Treatment Pad Placement.....	20
Pad Placement - Maintenance Treatment .....	21
LED Light Summary .....	22-23
#1 Basic Microcurrent Protocol.....	24-25
#2 Microcurrent with LED Light Therapy Protocol.....	26-28
Body Pad Placement.....	29-32
Mandatory Warranty Guidelines.....	33
Return Policy.....	34
Informed Consent ~ Sample Form.....	35
Microcurrent Suggested Informed Consent ~ Sample Form.....	36
LED Light Therapy Suggested Informed Consent ~ Sample Form.....	37
Skin for Life pure science professional skin care overview.....	38
Cost per treatment.....	39
Warranty Card.....	40
Customer Satisfaction Survey.....	41

*Please complete and return your Warranty Card to Skin for Life through any of the following methods:*

Fax: (352) 797.8669

Email: [sales@skinforlife.com](mailto:sales@skinforlife.com)

Mail to: 15241 Telcom Dr., Brooksville, FL 34604

# NUE FUSION 600 MICROCURRENT



## Important Safety Instructions

### Operation Safety Instructions –

- Read all instructions before use to avoid injury.
- To avoid serious injury or death, follow safety information in this document.
- Keep this manual for reference.
- Read all product safety labels.
- Refer to local building codes for installation requirements.
- Always turn off machine when not in use.

### Safety Information:

#### All Electrical Devices for Prevention of Electrical Shock Exposure -

- Keep machine unplugged when setting up machine to prevent accidental injury.
- Make sure machine is on a clean and dry surface area to prevent any accidental injury.
- Prohibit water from seeping into the machine housing.
- Avoid any water and/or liquids around the vicinity of the machine when in use or not in use to prevent any accidental injury.
- Check the cord and plug for damage before use.
- Pay attention to safety labels on machine before plugging the cord into an outlet.
- Switch on the equipment only after plugging into an outlet. Always turn off the machine before unplugging the cord from the outlet.
- Allow sufficient space for ventilation to prevent overheating.
- Avoid placing the cord near any hot objects.

# What is Microcurrent?

Small electrical pulses that trigger a firming and tightening response to skin. The skin is re-educated by microcurrent stimulus. This stimulation becomes a cumulative effect, thereby each procedure makes the skin's appearance smoother, firmer, and more resilient. Microcurrent gives tired, wrinkled, and dull skin a more youthful vibrant appearance.

Emil Y. Chi PhD studied the effects of microcurrent and found that after examination of the skin tissue treated by microcurrent, collagen thickness in connective tissue increased 10% and the number of blood vessels and circulation increased by 35%. Dr. Chi concluded that microcurrent to skin produced a firmer and tighter feeling to the skin surface.

Microcurrent is a great alternative to many invasive procedures, or you may use in conjunction with other cosmetic procedures to maintain a youthful appearance.

## Skin for Life Technology Nue Fusion Microcurrent

Skin for Life started manufacturing microcurrent equipment in 2003. We went through strict testing with our machines to verify each frequency, which assisted us with achieving approval for CE, an electrical safety standard that must be met before you can market equipment in Europe and other countries.

Our microcurrent machines are FDA Registered #1226143, which means we may sell to licensed professionals and physicians. It is our goal to provide professional quality, innovation, and excellent service, with the purchase of every Skin for Life Nue Fusion Microcurrent.

Our technology is based on programs from 1 - 10, total machine intensity from 75% - 100%, and individual intensity from 1 - 80, these variations in frequencies provide unlimited possibilities to help make a tremendous difference for mature skins. Our equipment is customizable, so you may have multiple possibilities for skin rejuvenation.

References: *Emil Y Chi, PhD, Director of Washington's Department of Pathology.*

# NUE FUSION 600 MICROCURRENT

## Features

- Non - invasive
- Progressive, not aggressive
- Results are cumulative
- Performed in a series for optimal results
- May be used in conjunction with other spa procedures

## Benefits

- Smooths fine line and wrinkles
- Diminishes expression aging
- Redefines neck, jowls, cheek, and eye areas
- Increases blood circulation
- Indirectly stimulates fibroblast

## Contra Indications

- Pace maker
- Pregnancy
- Melanoma
- Epilepsy
- Skin irritations
- Cancer
- Vericose Veins
- Phlebitus
- Herpes Zoster (Shingles)

## Precautions

- Diabetes
- Stroke
- Bell's palsy
- Rosacea
- Metal pins or plates
- Migraines
- Acne - Inflammation

*Skin for Life Nue Fusion Microcurrent Equipment is for esthetic use only, performed by licensed spa professionals, in accordance to cosmetology or esthetic state law requirements. No medical claims should be made, regarding Skin for Life Nue Fusion Microcurrent Equipment.*

# NUE FUSION 600 MICROCURRENT Functions


**Test Mode** - used to confirm current is flowing between wires.



Testing each of the ten lead cords is important before performing a microcurrent procedure. The “TEST Module” is in the back of the machine, where it indicates “TEST” (wire test port).

Plug lead cord into “wire test port”, then touch the two ends of the copper stems together and view the “wire test indicator light” (*under the power button and is marked “TEST” on the operation panel*) to make sure light is “green”. Green means a good working wire, if the light does not turn green then you have an impaired wire.

# NUE FUSION 600 MICROCURRENT Functions

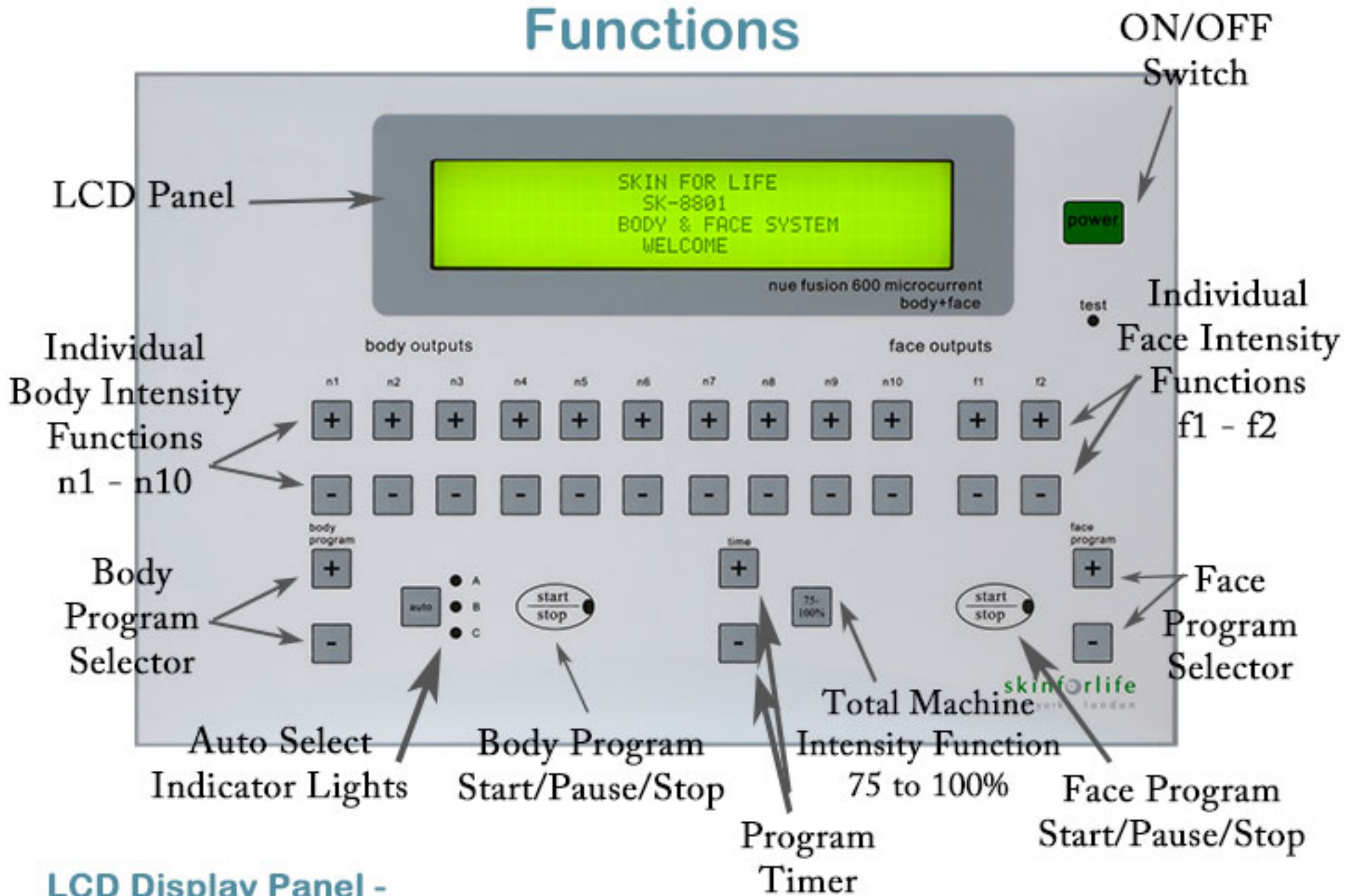

## LCD Display Panel -

**TIME = 30** - time defaults to 30 minutes, time may be increased up to 60 minutes.

**OUT = 75%** - Total Machine Intensity - may be increased in increments of 5% using the incremental intensity control.

**BODY/PROG = 1** - program settings are 1 - 10.

**FACE/PROG = 1** - program settings are 1 - 10.

**N1 - N10** - Body Outputs - Defaults to "0". This is the individual lead/output intensity range 1 - 80.

**F1 - F2** - Face Outputs - Defaults to "0", individual lead/output intensity range 1 - 80.

**START/PAUSE/STOP** - you may start your machine after setting your program and total machine intensity. Make sure N1 - N10 are at "0". When pausing or stopping your machine, make sure N1 - N10 are at "0". When performing with F1 or F2, make sure individual intensity are set at "0". When pausing or stopping your machine, make sure F1 - F2 are at "0".

**AUTO FACIAL MAINTENANCE** - randomly cycles through each program. This program is recommended for maintenance treatments for pad use only.





# NUE FUSION 600 MICROCURRENT

## Procedure & Techniques

### Setting Your Microcurrent for a Facial Procedure -

Recommended twice a week for at least three weeks and maximum twice a week for six weeks. Complete movements to one side at a time.

### First Step

**Program Setting - Base Energy - Adjust for client/patient comfort**

Program 3 - 4	Neck and Jowls	use Ball Wands
Program 5 - 6 - 7	Jaw line and Cheeks	use Ball Wands
Programs 8 - 9	Eyes and Forehead	use Ball Wands

### Second Step

**Total Machine Intensity - Intense Energy - Adjust for client/patient comfort**

80% (+/- 5%) - Neck and Jowls

80% (+/- 5%) - Jaw line and Cheeks

75% - 80% (+/- 5%) - Eyes and Forehead

**Push Start Button to initiate the flow of current after selecting Program and Total Machine Intensity. Green Flashing Light Indicates the Machine is Working and You May Begin to Select Your Individual Intensity.**

### Third Step

**Individual Intensity - Increases the Programs Intensity -  
Adjust for client/patient comfort**

35 (+/- 5) - Neck and Jowls

35 (+/- 5) - Jaw line and Cheeks

30 (+/- 5) - Eyes and Forehead

# NUE FUSION 600 MICROCURRENT Procedure & Techniques

After completing all of the facial areas with the “Ball Wands” now transition to Straight Wands for the Final Step.

## Skin Tightening - Free Style Techniques for Face and Neck -

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 10	Toning / Tightening Face and Neck	Straight Wands	75%	25 (+/- 5)


**NOTE\*\*\*** Only setting that may be adjusted is the “Individual Intensity”.

You may work horizontally, vertically, and diagonally for lifting, tightening, and toning the skin. Be consistent with each side of the face and neck with performing identical movements to both sides.

Procedural Tips - Always keep the “**RED**” lead above the “**BLACK**”, and always have “**BLACK**” moving towards “**RED**” when using Ball Wands and Straight Wands.


For best results each stroke or movement should be repeated at least 3 times with increasing individual frequency and/or intensity with each visit, however not every client/patient will be able to **tolerate** increased intensity and/or frequency. This may take several visits, with gradually increasing frequencies. However, always keep in mind your client/patient’s comfort.

# Facial Muscles

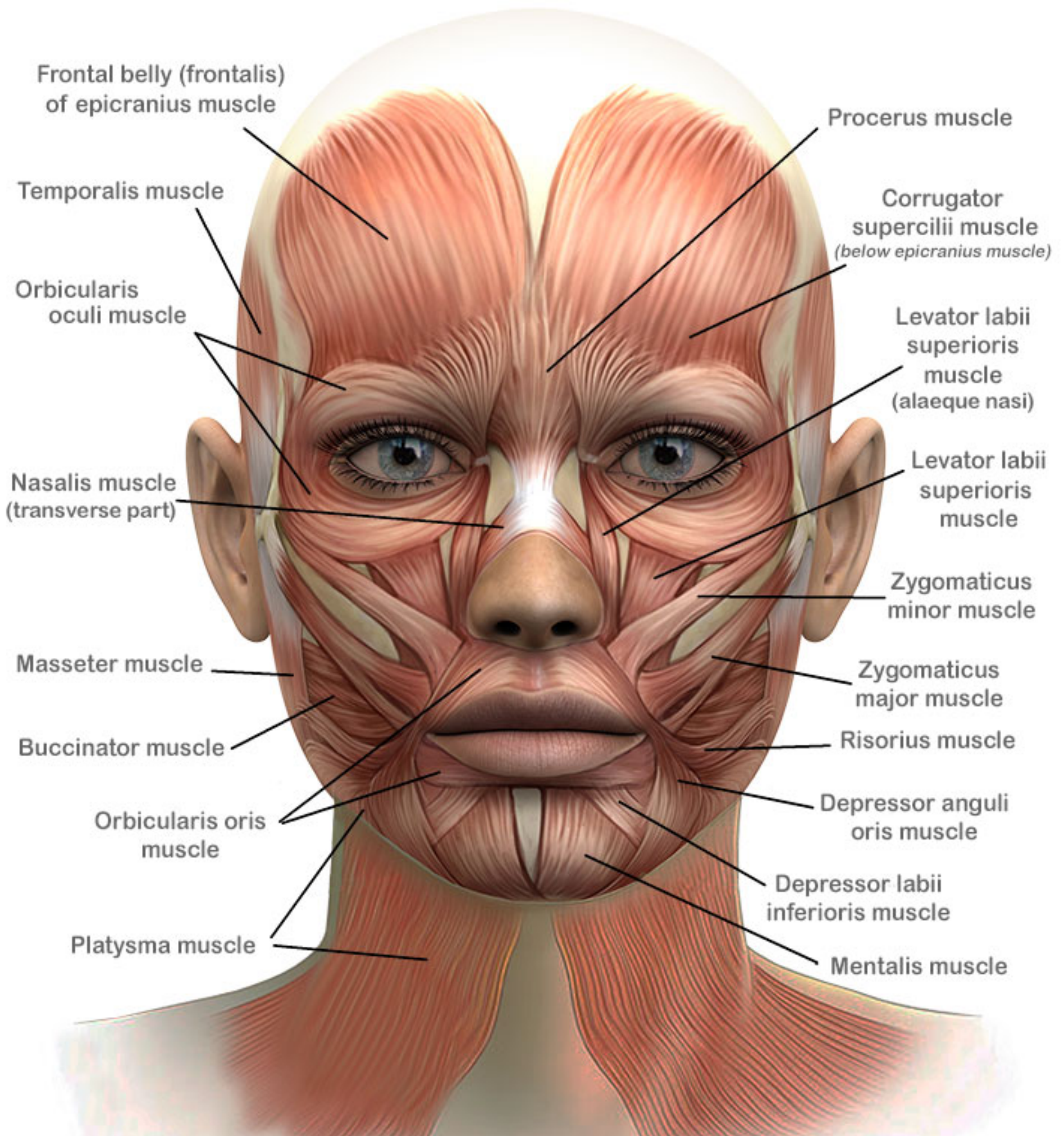

# NUE FUSION 600 MICROCURRENT

## Microcurrent Suggested Facial Movements

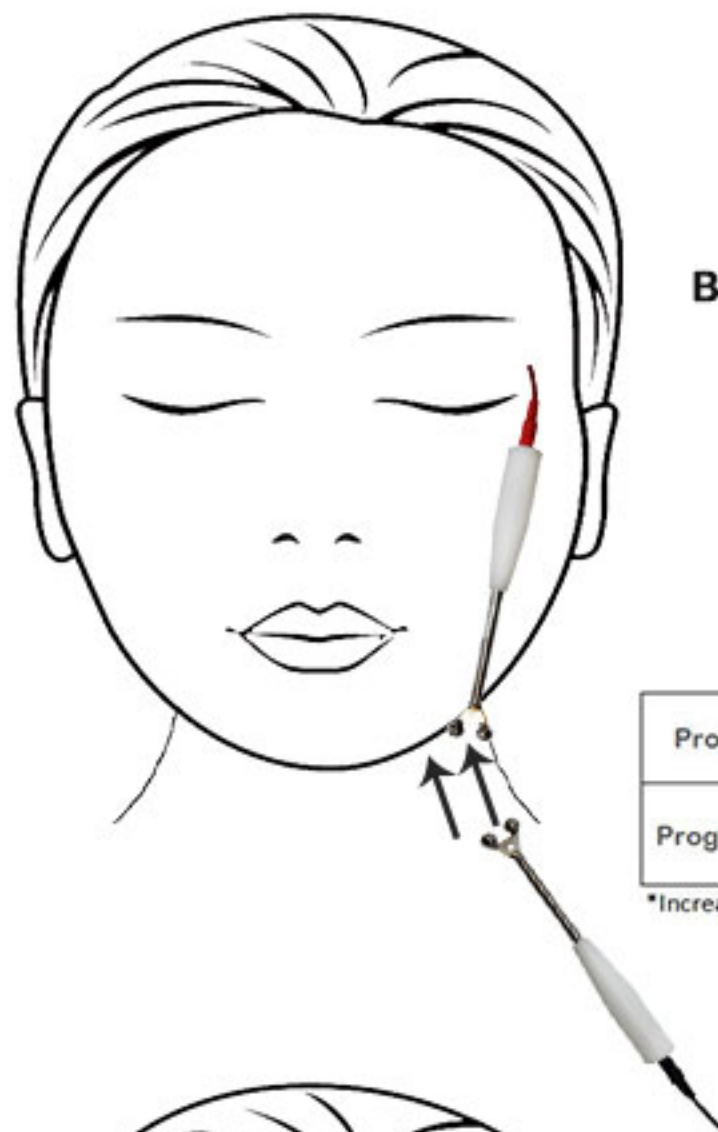

### Platysma 1

This muscle is from the clavicle to jaw line. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.


**Red Probe** is stationary on the outer jaw line and the **Black Probe** is positioned on the lateral part of the clavicle.

Movements of 3 repetitions with counts of 4 - 5 seconds with each pass.

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 3	Neck and jowls	Ball Wands	80%	35 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

Arrows Point to the Direction of Movement.

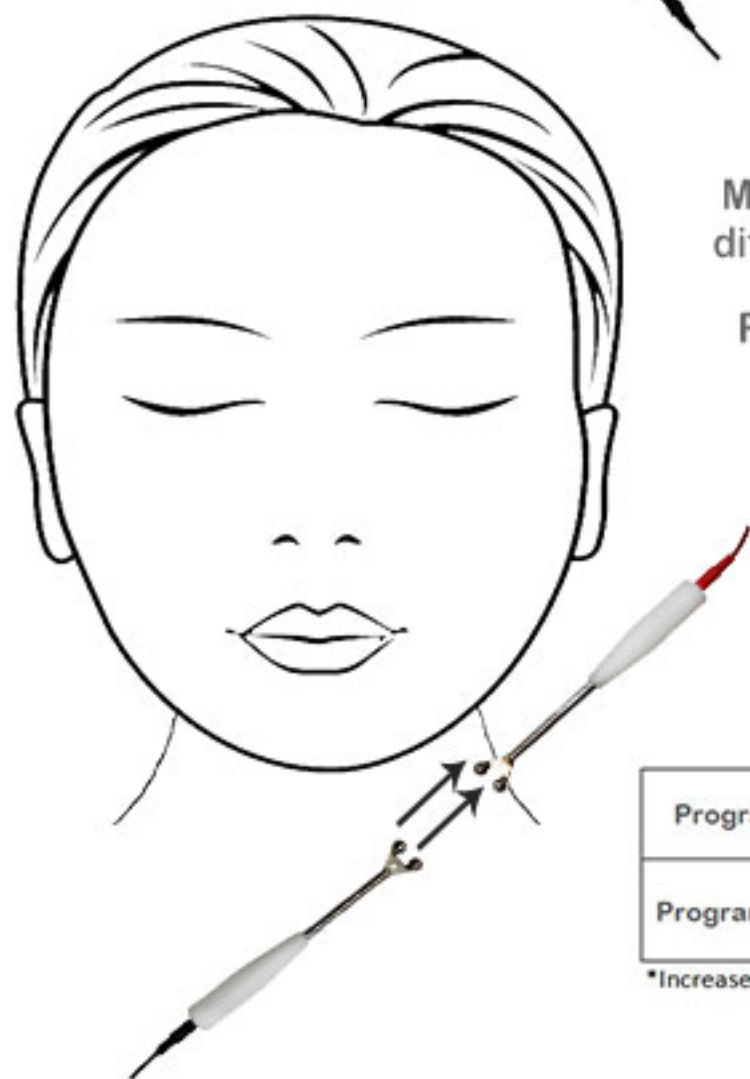

### Platysma 2

Muscle is striated, so you want to work the muscle in different angles to excite the response of stimulation.

Place the **Red Probe** on the lateral jaw line and the **Black Probe** on the medial line of the clavicle.

The **Black Probe** moves outwards towards the **Red Probe**. Movements of 3 repetitions with counts of 4-5 seconds with each pass.

**\*DO NOT go behind the SCM (Sterno Cleido Mastoid)**

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 3	Neck and jowls	Ball Wands	80%	35 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

Arrows Point to the Direction of Movement.

# NUE FUSION 600 MICROCURRENT

## Microcurrent Suggested Facial Movements...

### Sub Mentalis Horizontal Firming Movements



Place **Red Probe** on the lateral R. Depressor labii inferioris and the **Black Probe** on the sub mental area (*under the Mentalis Muscle*) move toward the **Red Probe**, movements of 3 repetitions with counts of 4 - 5 seconds.

Once completed reverse your probes and move the opposite direction. This works on double chins or loose skin to firm and tighten.

#### Starting Point - neck and jowls

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 3	Neck and jowls	Ball Wands	80%	35 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

(Change Settings for Next Areas)

### Mandibular and Zygomaticus Lifting and Holding Diagonal Movements



Place **Red Probe** anterior of the medial auricular (*middle ear*) and the **Black Probe** starting at the base of the masseter and move diagonally in the direction of the arrow.

Hold each stroke for 3 - 5 seconds for lifting and holding and perform 3 repetitions.

#### Starting Point - jaw line and cheeks


Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 5	Jaw line and cheeks	Ball Wands	80%	35 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

Arrows Point to the Direction of Movement.

# NUE FUSION 600 MICROCURRENT

## Microcurrent Suggested Facial Movements...

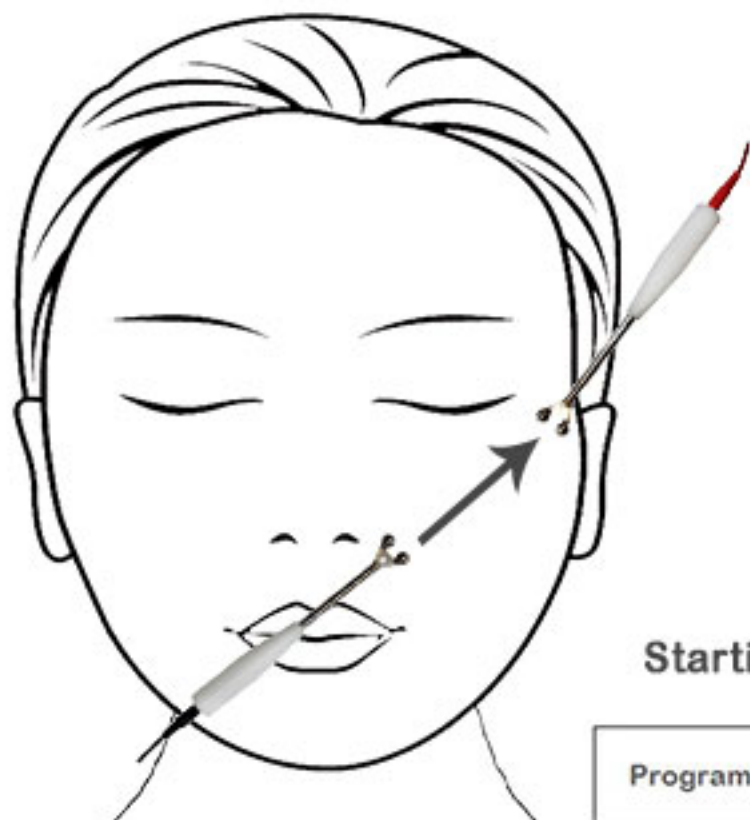

### Zygomaticus Major and Minor Diagonal Lift and Hold Movements

Place **Red Probe** to the upper lateral part of the zygomatic muscle and the **Black Probe** to the medial area of the zygomatic muscle. Move **Black Probe** toward **Red Probe** while lifting and holding for 3 - 5 seconds and continue for 3 repetitions.


This is to strengthen the zygomaticus area while lifting and smoothing the nasal labial lines.

#### Starting Point - jaw line and cheeks

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 5	Jaw line and cheeks	Ball Wands	80%	35 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

*(Change Settings for Eyes & Forehead Areas)*

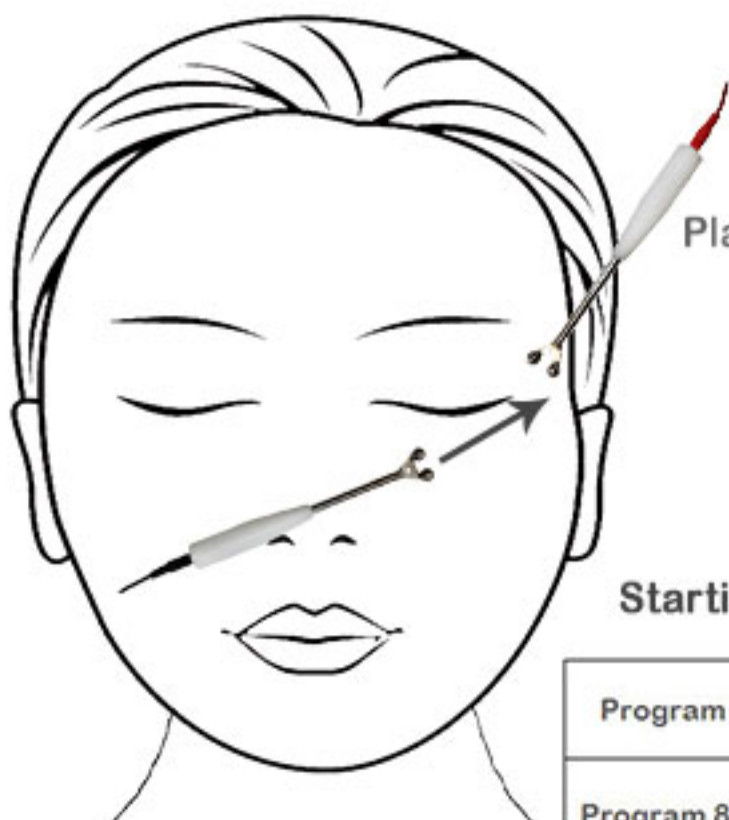

### Obicularis Oculi Muscle 1 Firm and Tighten the Eye Area

Place **Red Probe** at the mid line area of the infra orbital bone and **Black Probe** next to the nose and move toward the **Red Probe** for 3 repetitions and 2 - 3 seconds with each stroke.

Continue the movements and transition to the temple area.

#### Starting Point - eyes and forehead


Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 8	Eyes and forehead	Ball Wands	75% - 80%	25 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

Move **Black Probe** towards **Red Probe**  
Arrows point in the direction of movement.

# NUE FUSION 600 MICROCURRENT

## Microcurrent Suggested Facial Movements...

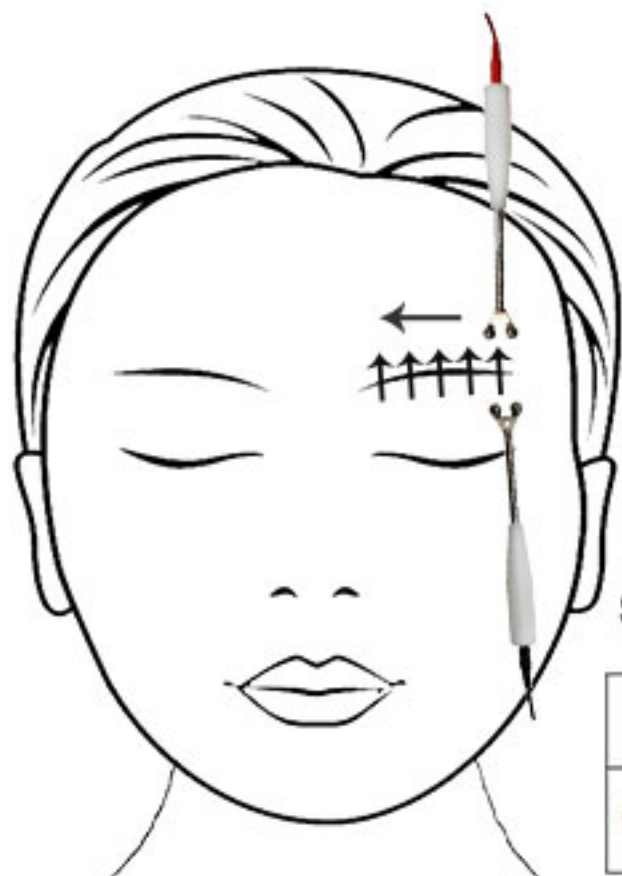

### Obicularis Oculi Muscle 2 Lift and Hold

Place the **Red Probe** on the lateral side of the supra orbital bone and place the **Black Probe** inferior to the **Red Probe** and lift **Black Probe** and hold 3 - 5 seconds and continue moving across for 3 repetitions.

This movement is to open the eye vertically.

**Starting Point - eyes and forehead**

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 8	Eyes and forehead	Ball Wands	75% - 80%	25 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.



### Frontalis Muscle Firm + Lift

Place the **Red Probe** to the lateral part of the frontalis muscle and place the **Black Probe** at the anterior of the temple moving toward the **Red Probe**.

Transition after 3 repetitions and with the count of 3 - 5 seconds per stroke.

**Starting Point - eyes and forehead**

Program	Area	Wand Type	Total Machine Intensity	Individual Intensity (increments)
Program 8	Eyes and forehead	Ball Wands	75% - 80%	25 (+/- 5)

\*Increase individual intensity in increments of 5 when necessary.

Move **Black Probe** towards **Red Probe**  
Arrows point to steps in the direction of movement.

# NUE FUSION 600 MICROCURRENT

## Skin Tightening and Firming with Straight Wands



### Platysma

This muscle is from the clavicle to jaw line. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the outer jaw line and the **Black Probe** is positioned on the lateral part of the clavicle.

Movements of 3 repetitions with counts of 4 - 5 seconds with each pass.

You may **only** increase the "Individual intensity" in increments of 5.



### Sub Mental Area

This muscle is from one side of sub mental to the other side. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the depressor aguli oris and the **Black Probe** is positioned on the sub mental area.

Movements of 3 repetitions with counts of 4 - 5 seconds with each pass.

Switch probes and go in the other direction to strengthen the band of muscle.


This is sometimes referred to as the "Turkey Waddle" - to strengthen and firm.

You may **only** increase "Individual intensity" in increments of 5.



# NUE FUSION 600 MICROCURRENT

## Skin Tightening and Firming with Straight Wands

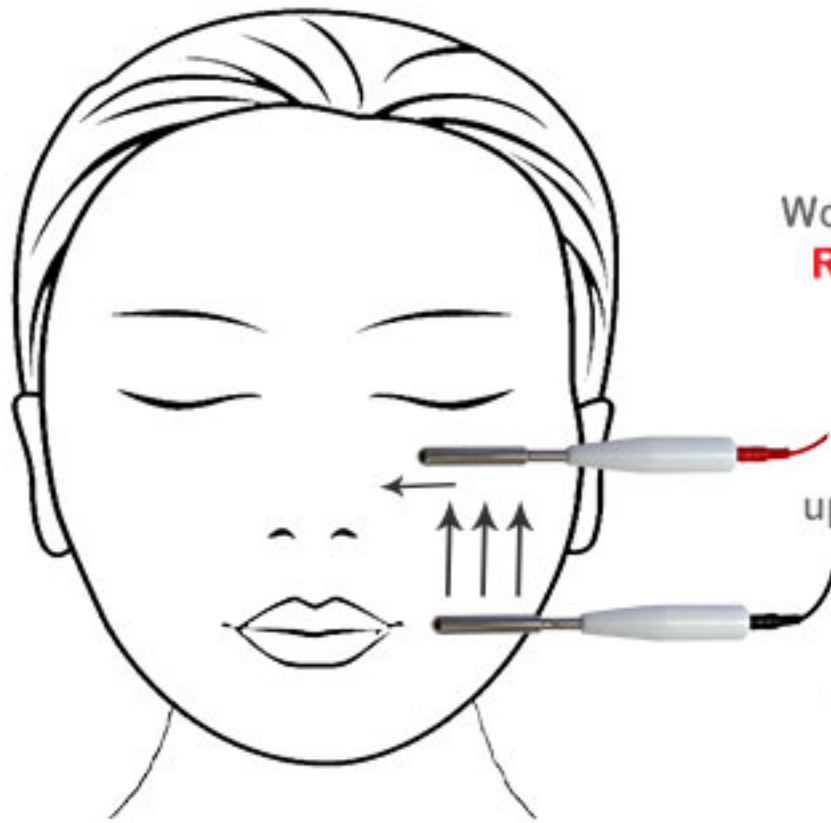

### Cheek Area Masseter - Zygomatic Area

Working from jaw line to upper cheek area. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the upper zygomatic bone and the **Black Probe** is positioned on the lateral part jawline.

Movements of 3 repetitions with counts of 4 - 5 seconds with each pass. Move across towards the nose with lifting movements.

You may **only** increase the "Individual intensity" in increments of 5.

Arrows Point to the Direction of Movement.



### Cheek Area - Zygomatic Muscle 2

Working from the corner of the mouth to temporal bone. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the corner of the mouth and the **Black Probe** is positioned on the temporal bone.


Movements of 3 repetitions with counts of 4 - 5 seconds with each pass.

Movements are toward the temporal bone. Lift and hold belly of muscle and smooth toward the temporal bone.

You may **only** increase "Individual intensity" in increments of 5.

# NUE FUSION 600 MICROCURRENT

## Skin Tightening and Firming with Straight Wands

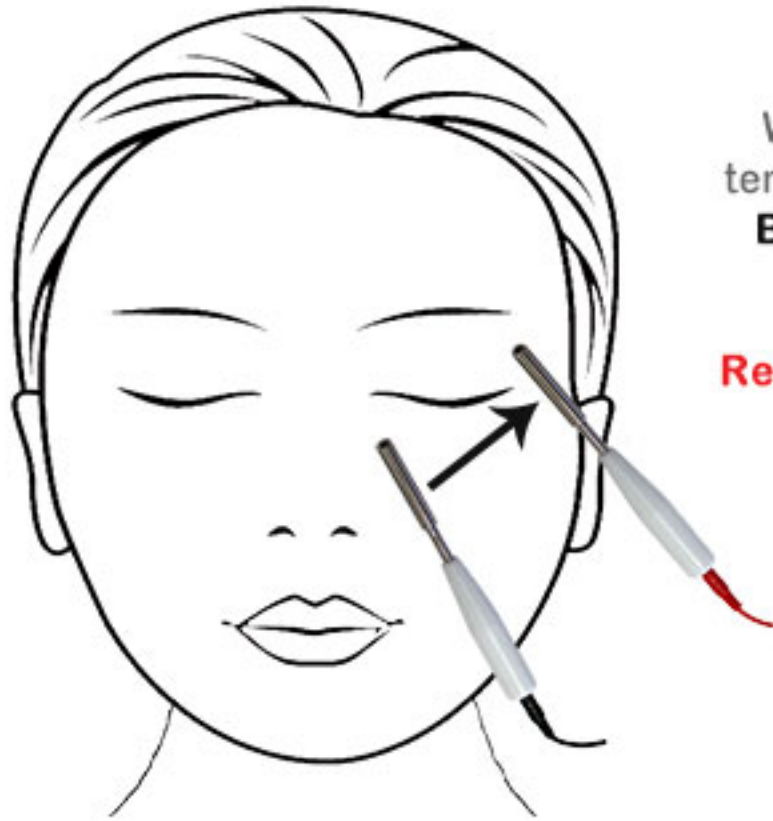


### Eye Area - Obicularius Oculi 1

Working from the midline inferior orbital bone to temporal bone. Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the upper temporal bone and the **Black Probe** is positioned on the inner inferior orbital.

Movements of 3 repetitions with counts of 4 - 5 seconds with each pass. Movements are toward the temporal bone.

You may **only** increase the "Individual intensity" in increments of 5.

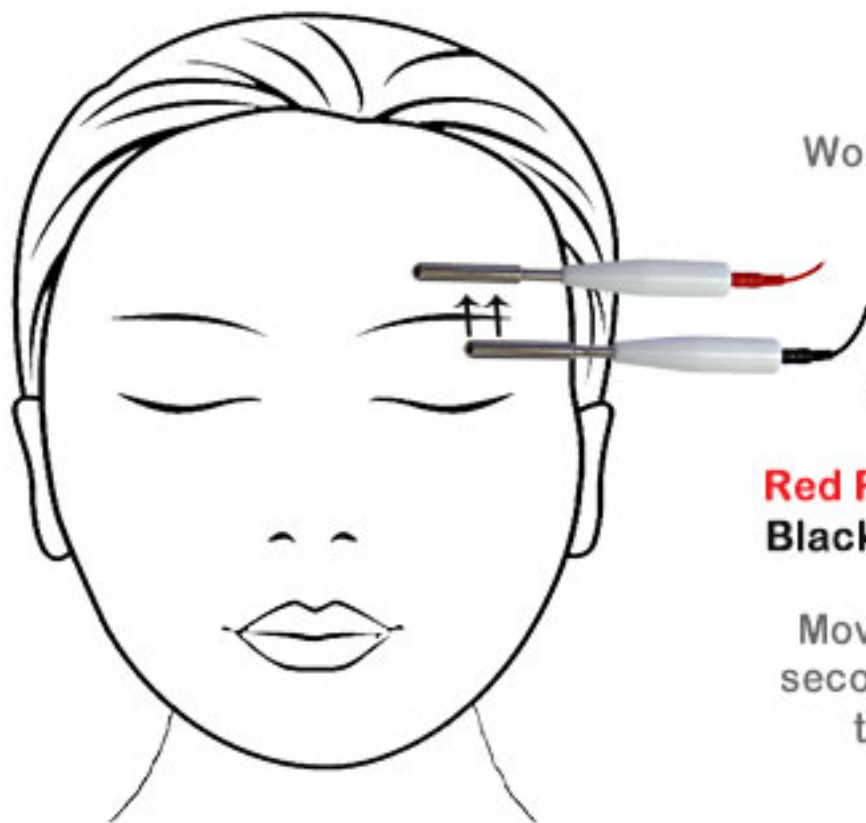

### Brow Bone - Supra Orbital

Working from the lateral supra orbital bone to the mid line of procerus muscle.

Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary above brow bone and the **Black Probe** is positioned below the orbital bone.


Movements of 3 repetitions with counts of 3 - 5 seconds with each pass. Movements are toward the procerus muscle. Lifting and holding movements as you move across.

You may **only** increase "Individual intensity" in increments of 5.

# NUE FUSION 600 MICROCURRENT

## Skin Tightening and Firming with Straight Wands

### Forehead - Frontalis

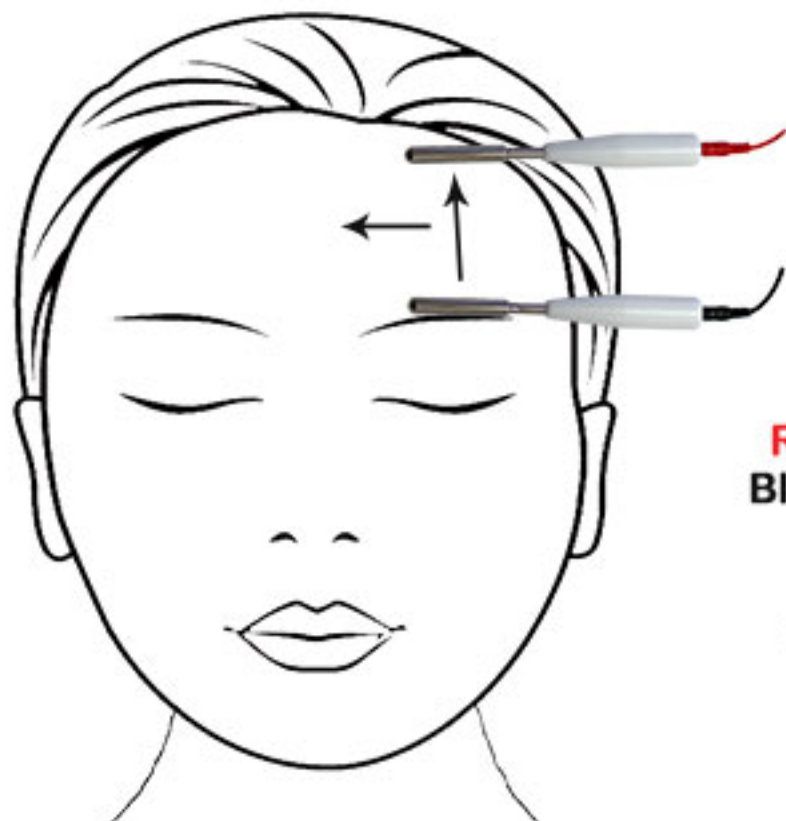

Working from the lateral supra orbital bone to the midline of the forehead.

Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the hair line and the **Black Probe** is positioned above the orbital bone.

Movements of 3 repetitions with counts of 3 - 5 seconds with each pass. Movements are toward the midline of the forehead. Lifting movements as you move across.

You may **only** increase the "Individual intensity" in increments of 5.

### Forehead - Frontalis



Working from the procerus to the lateral of the forehead.

Your **Red Probe** is always above the **Black Probe**. The **Black Probe** is always moving toward the **Red Probe**.

**Red Probe** is stationary on the hair line and the **Black Probe** is positioned on the procerus.

Movements of 3 repetitions with counts of 3 - 5 seconds with each pass. Diagonal Movements are toward the midline of the forehead. Lifting movements as you move across.


You may **only** increase "Individual intensity" in increments of 5.

Now that you have completed one side **REPEAT**  
"Ball and Straight Wands" to the other side of the face and neck.

# NUE FUSION 600 MICROCURRENT Facial Maintenance Treatment Pad Placement

Apply pads at least 1 inch apart. If pads touch they will become neutral.

Below is pad placement which indicates maintenance treatment using 30R facial pads, (48S facial pads may be used on the neck only depending on the space available).

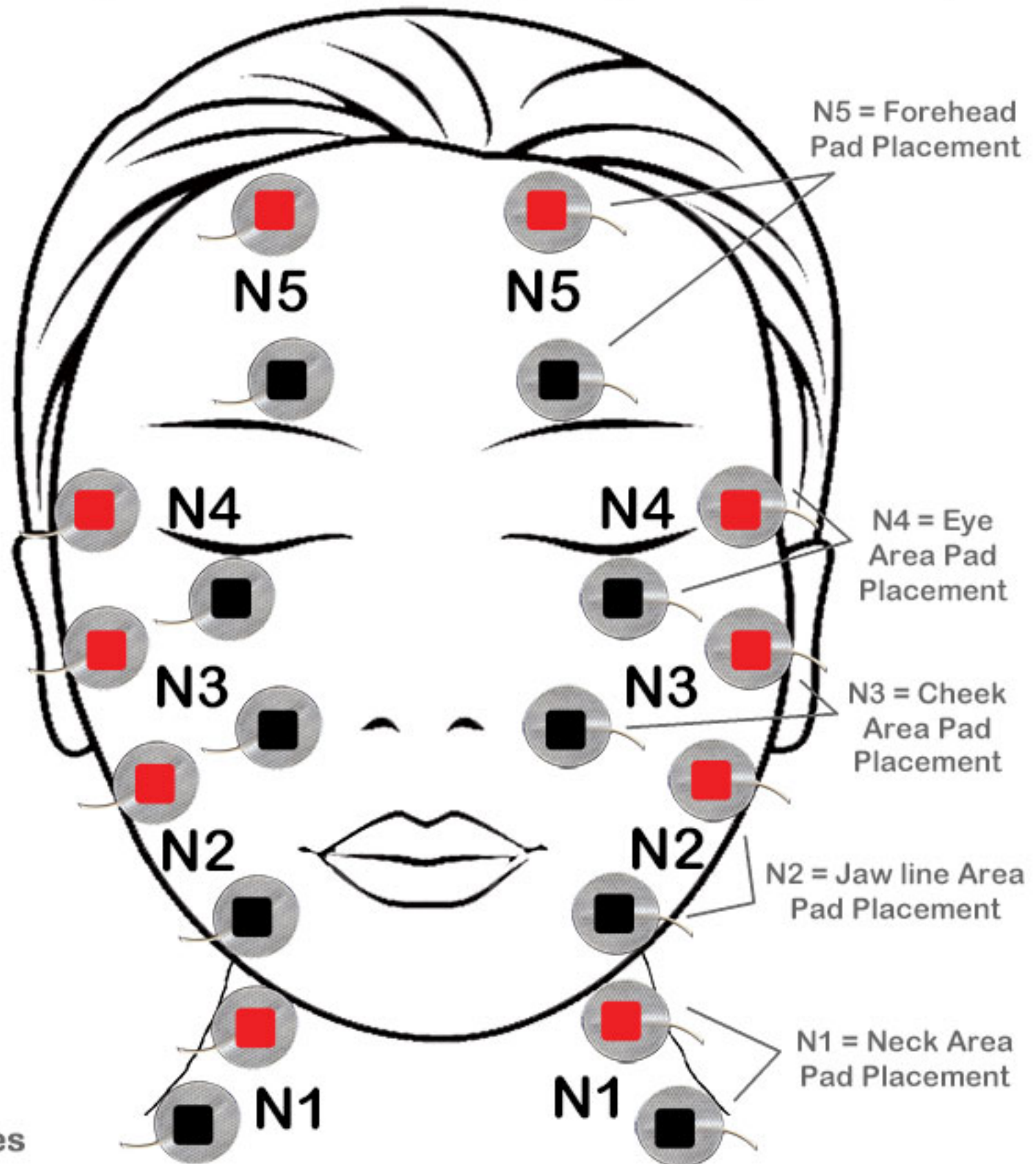

**20 Minutes**

When transitioning, decrease "ALL" individual intensity lead cords to "0" before removing the pads. Once pads are removed, apply to the other side of the face and neck. Please refer to "Pad Placement - Tightening and Toning Skin" for additional references.

# NUE FUSION 600 MICROCURRENT

## Tightening and Toning using Pad Placement

### - Maintenance Treatment

Turn on your machine per instructions on page 10. Pay Attention to safety instructions.

Insert all 10 lead cords into the front of the machine. Attach pads to the lead cords - it does not matter which pads are attached. All pads are the same.


Place Pads using the diagram on page 20 - make sure your individual intensity buttons are at "0" - this means there is no energy coming through the lead cords.

Program your machine with the following settings -

Program - 10

Total Machine Intensity - 75%

Turn On "Auto Facial Maintenance" - each A, B, and C will cycle with various programs to strengthen skin response. Make sure all individual intensity buttons are set at "0".

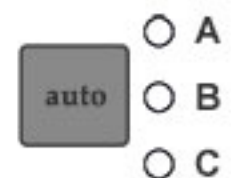

**Start with the following -**

N1 lead cord - increase the N1 button to a comfortable setting for your client. Your client will feel a fast twitch, you do not want your client uncomfortable. More is not better.

N2 lead cord - increase the N2 button to a comfortable setting for your client. Fast twitch response.

N3 lead cord - increase the N3 button to a comfortable setting for your client. Repeat same steps as above.

N4 lead cord - increase the N4 button to a comfortable setting for your client. Repeat same steps as above.

N5 lead cord - increase the N5 button to a comfortable setting for your client. Repeat same steps as above.

N1 - N5 is for one side of the face and neck as indicated on the diagram.

Complete N6 - N10 to the other side of face and neck, mimicking the same pad placement as the other side as N1 - N5.

Set your "TIME" to 20 minutes and try to adjust your individual intensity in increments of "5" with every 5 minutes. However, if your client is comfortable with the settings do not increase.

Once your microcurrent treatment is completed, make sure you decrease "ALL" lead cords N1 - N10 buttons to "0" then "TURN OFF" the machine. Start removing the pads one at a time and store on the same plastic card and put into a plastic zip locked bag with your clients name on the bag. **"DO NOT SHARE PADS"** with other clients/patients.

Maintenance Microcurrent Treatments are minimum once a month and maximum twice a month for sustainable results.


# LED Light Summary

LED technology was originally developed to enhance the growth of plant tissue in space by NASA's Marshall Space Flight Center and Quantum Devices, Inc. of Barneveld, Wisconsin. LED's have a similar physiological effect on human cells as they do on plant cells. It is a natural photo - biochemical reaction that is similar to photosynthesis in plants, which triggers the body to convert light energy into cell energy.

Age defying - 680nm  
(RED)

Oily/Troubled Skin - 460nm  
(BLUE)

Skin Rebuilding - 800nm  
(Infrared)

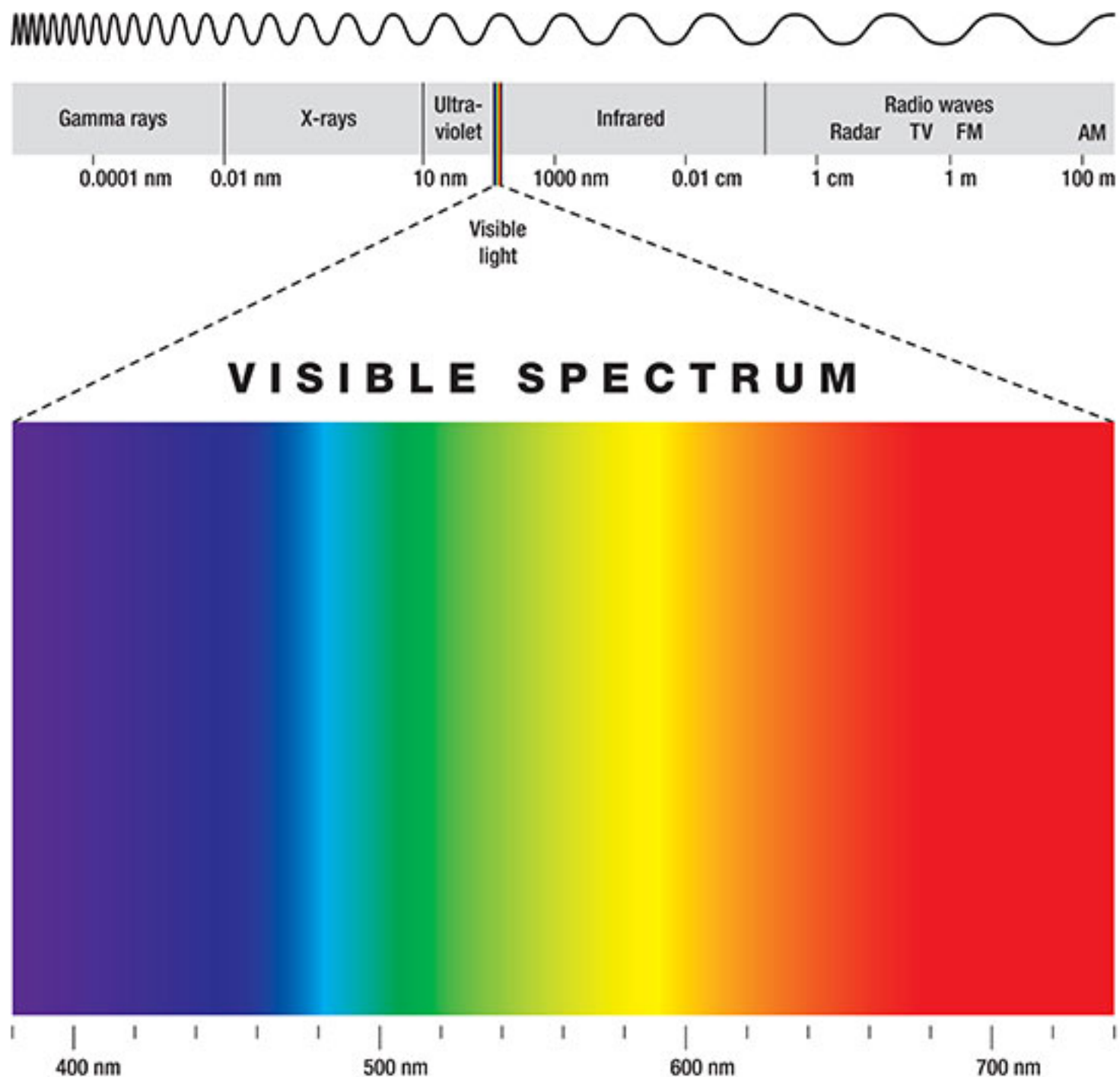

## LED Light Summary - continued...

This remarkable technology, enhances the ATP (Adenosine Tri phosphate) within the mitochondria of the cells which excites the first process of cell division. This non thermal response relies on the visible light spectrum to interact with cells for a growth response.



**RED LED Light** is designed to trigger the first initial ATP response. RED LED Light should always be the first light used on all skin types, conditions, and colors.

**Skin Indications** – sun damaged skin, mature, pigmented lesions, oily, troubled skin, capillary distention, and temporary/chronic sensitive skins.

**How To Use** – RED hand piece – use 90 seconds per area, with overlapping 30% - 50% completing the process within the process within 12 – 15 minutes before transitioning to another LED Light color.

**RED LED Light Stand Alone Standard Head Unit**

– use 20 minutes before transitioning to another LED Light color.



**BLUE LED Light** is suggested for oily and troubled skin conditions. The depth of this light is important to minimize the activity of bacteria within the sebaceous gland that is associated with blemished skins.

**Skin Indications** – oily, troubled skin, blemished skin, and seborrhea skins.

**How To Use** – BLUE LED Light hand piece for 90 seconds per area with overlapping 30% - 50% completing the process in 10 minutes.

**BLUE LED Light Stand Alone Unit** – use 10 – 15 minutes after the RED LED Light stand-alone unit.



**GREEN LED Light** is recommended for pigmented lesions, like melasma, chloasma, and solar lentigines. **Skin Indications** – discolored skin, dark spots, age spots, pregnancy mask, melasma. **How To Use** – use 10 – 15 minutes after the RED LED Light Stand Alone unit. *(NO hand pieces available)*

**YELLOW LED Light** assist lymphatic drainage, with edema and erythema conditions. **Skin Indications** – swelling, edema, redness, erythema. **How To Use** – use 10 – 15 minutes after RED LED Light Stand Alone unit. *(NO hand pieces available)*

# NUE FUSION 600 MICROCURRENT

## #1 Basic Facial Microcurrent Protocol

**Cleanse ~ Gentle Milk Cleanser + vitamins A & E** - Apply to skin with fluid movements for a thorough cleansing. Remove with warm towel, sponges, and/or gauze.

**Tone ~ Instant Soothing Toner + natural actives** – Apply to face and neck after you have removed the Gentle Milk Cleanser + vitamins A & E. This prepares the skin for the next step. Dry skin thoroughly.

**Exfoliation and/or selective add on applications that may be used –**

- Microdermabrasion – use (1) pass application over the entire face and neck.
- Enzyme Exfoliation – apply per product directions.
- Lactic Acid Cosmetic Peel – one layer application and neutralize thoroughly.
- Oxygen Infusion – to enhance the vitality of skin health prior to microcurrent.
- Dermaplaning – use (1) pass application.

Remove any residue with warm towel (*not hot*). Remove thoroughly.

**Tone ~ Instant Soothing Toner + natural actives** – Apply to face and neck prior to serums and moisturizers.

**Serums** – apply Vita C Peptide serum + LSS™ delivery and Moisture Booster + hyaluronic acid with LSS™ delivery to face and neck. Building skin strength starts with the change every skin needs from performance serums.

**Moisturizers** – apply Vital Recovery Concentrate + natural botanicals with LSS™ delivery and Ultra Performing Complex + A, C, E complex to face and neck. Protecting the skin with extra nourishment for superior hydration.

**MASSAGE** – add your best massage techniques for 5 – 7 minutes to invigorate the skin and as a great start for toning and lifting the skin for microcurrent.

**Conductivity Gel** - apply a medium thick layer to the first area of treatment.

*Please review the movements from previous pages titled “Microcurrent Techniques”.*

After your conductivity application, saturate a facial mask brush with water and apply over the conductivity gel for added moisture.

**Perform Microcurrent Techniques** – refer to the “Suggested Facial Movements” section in Manual for additional details.

- Ball Wands – first application for skin strengthening (firming/lifting).
- Straight Wands – second application for skin toning

**Once Microcurrent Treatment is completed  
continue with steps on next page...**



# NUE FUSION 600 MICROCURRENT

## #1 Basic Facial Microcurrent Protocol... *continued*

Remove conductivity gel with a warm towel (*not hot towel*). Remove thoroughly.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face, neck, and chest to prepare the skin for the next step of treatment. Dry skin thoroughly.

**Active + Restore** – apply to face and neck and blend well.

- Vita C Peptide Serum with LSS™ delivery
- Moisture Booster + hyaluronic acid with LSS™ delivery

**Nourish + Protect**

- Vital Recovery Concentrate + botanical lipids with LSS™ delivery
- Ultra Performing Complex + A, C, E complex with LSS™ delivery
- Moisture Complex + nourishing peptides with LSS™ delivery

**Algae Vitamin Mask phase 1 + Algae Liquid Booster phase 2 –**

Step 1: Empty Algae Vitamin Mask phase 1 into a glass or stainless bowl.

Step 2: Empty the content of Algae Booster phase 2 into the same bowl.

Step 3: Mix together step 1 and step 2 until you obtain a smooth consistency. You will need to be quick with your stirring.

Step 4: Start with the forehead and transition to other parts of the face, be thorough. Think about frosting a cake, same concept. If you want to apply mask to neck area you will need to use an additional mask sachet, phase 1 and phase 2.

The mask will rubberize after 5 – 10 minutes and then you may lift off to remove the mask. Use sponges, gauze, or a cool moist towel to remove.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck to remove mask residue.

**Active + Restore**

- Vita C Peptide Serum with LSS™ delivery

**Nourish + Protect**

- Vital Recovery Concentrate + botanical lipids with LSS™ delivery

**Physical Protection**

- SOL defense SPF 30 Broad Spectrum Sunscreen

# NUE FUSION 600 MICROCURRENT

## #2 Microcurrent with LED Light Therapy

### Skin Fitness Deluxe Program (*Face only*)

**Cleanse ~ Gentle Milk Cleanser + vitamins A & E** - Apply to skin with fluid movements for a thorough cleansing. Remove with warm towel, sponges, and/or gauze.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck after you have removed the Gentle Milk Cleanser. This prepares the skin for the next step. Dry skin thoroughly.

**Exfoliation** - You may use (1) type of exfoliation or many as your building Skin Fitness Treatment.

- Microdermabrasion –
  - o (1) pass application with aluminum oxides
  - o (1) pass application with organic grains
- Exfoliate Enzyme + papaya complex – see Skin for Life pure science professional brochure for instructions.
- Professional Peel System – see Skin for Life pure science professional brochure for instructions.

Remove with warm towel.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck to remove any residue from exfoliation and it is an excellent cool down before the next step.

**Oxygen Infusion** –

- Oxygen Infusion Gel phase 1 + A, C, E complex with LSS™ delivery – apply a medium thick layer to the entire face and neck. Manipulate with moderate massage movements concentrating on lifting movements. Manipulate the skin for 1 – 2 minutes. Leave on.
- Oxygen Infusion phase 2 + activator – put ½ cup into your airbrush gun. Start with the forehead where the tip of the airbrush gun is very close to the skin. Manipulate the gun by using circular movements. Perform at least (3) passes over the entire face and neck.
- Once complete manipulate with moderate massage movements concentrating on lifting movements. Manipulate the skin for 1 – 2 movements.

Remove with a warm towel (*not hot*). Remove thoroughly.

continue with steps on next page...

# NUE FUSION 600 MICROCURRENT

## #2 Microcurrent with LED Light Therapy

### Skin Fitness Deluxe Program (*Face only*)... *continued*

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck after you have removed the Gentle Milk Cleanser. This prepares the skin for the next step. Dry skin thoroughly.

**MASSAGE** – Apply the following Skin for Life pure science skin care products below.

- **Active + Restore:** Vita C Peptide Serum with LSS™ delivery and Moisture Booster + hyaluronic acid with LSS™ delivery – apply as a serum booster duo.
- **Nourish + Protect:** Ultra Performing Complex + A, C, E complex and Moisture Complex + nourishing complex. Mix equal parts and use as your massage medium for a super moisture enricher supplement.
- **Massage** by using your best techniques. Lifting and firming movements (pettrissage and tapotement).
- **Do Not Forget the “EYES”.**

**Conductivity Gel** - apply a medium thick layer to the first area of treatment. Please review the movements from previous pages titled “Microcurrent Techniques”. After your conductivity application saturate a facial mask brush with water and apply over the conductivity gel for added moisture.

**Perform Microcurrent Techniques** – See Manual for Additional Details.

- **Ball Wands** – first application for skin strengthening (firming/lifting).
- **Straight Wands** – second application for skin toning

Once Microcurrent Treatment is completed follow the below steps –

Remove conductivity gel with a warm towel not hot towel. Remove thoroughly.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck to remove any residue and to cool the skin. Dry Skin Completely.

**LED Light Hand Pieces** – Make sure you have “Eye Protection” before applying the first light.

**RED LED Light hand piece** – primary light for regeneration and stimulation for a cellular response. Areas for Treatment: sun damage, wrinkles, fine lines, and skin concerns (rosacea).

- Apply to the forehead first, keep the RED LED Light hand piece stationary and hold for 90 seconds.
- Repeat this process until you have performed the complete treatment to the face.
- Treatment time is minimum 12 minutes and maximum 15 minutes for an effective treatment.

**continue with steps on next page...**

# NUE FUSION 600 MICROCURRENT

## #2 Microcurrent with LED Light Therapy

### Skin Fitness Deluxe Program (*Face only*)... *continued*

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck to remove any residue and to cool the skin. Dry Skin Completely.

**Active + Restore** – apply to face and neck and blend well.

- Vita C Peptide Serum with LSS™ delivery
- Moisture Booster + hyaluronic acid with LSS™ delivery

**Nourish + Protect**

- Vital Recovery Concentrate + botanical lipids with LSS™ delivery
- Ultra Performing Complex + A, C, E complex with LSS™ delivery
- Moisture Complex + nourishing peptides with LSS™ delivery

**Algae Vitamin Mask phase 1 + Algae Liquid Booster phase 2 –**

Step 1: Empty Algae Vitamin Mask phase 1 into a glass or stainless bowl.

Step 2: Empty the content of Algae Booster phase 2 into the same bowl.

Step 3: Mix together step 1 and step 2 until you obtain a smooth consistency. You will need to be quick with your stirring.

Step 4: Start with the forehead and transition to other parts of the face, be thorough. Think about frosting a cake, same concept. If you want to apply mask to neck area you will need to use an additional mask sachet, phase 1 and phase 2.

The mask will rubberize after 5 – 10 minutes and then you may lift off to remove the mask. Use sponges, gauze, or a cool moist towel to remove.

**Tone ~ Instant Soothing Toner + natural actives** - Apply to face and neck to remove mask residue.

**Active + Restore**

- Vita C Peptide Serum with LSS™ delivery

**Nourish + Protect**

- Vital Recovery Concentrate + botanical lipids with LSS™ delivery

**Physical Protection**

- SOL defense SPF 30 Broad Spectrum Sunscreen

# NUE FUSION 600 MICROCURRENT

## Body Muscle Lift - Pad Placement


### Abdomen / Thighs / Buttocks Muscle Lift

Recommended treatments: 2x a week for 4-6 weeks or until desired results are achieved with maintenance of 2x a month.

	Body Type	Program	Overall Intensity	Individual Intensity
<b>Phase 1: fat reduction</b> 5 minutes	All	10*	75%	20+
<b>Phase 2: muscle lift</b> 20-30 minutes	Large/heavy	1-2	75% (+5)	50+
	Med/average	2-3	80% (+/-5)	40+
	Small/thin	3-4	80% (+/-5)	35+
<b>Phase 3: toning</b> 10-15 minutes	Large/heavy	4-6	75% (+/-5)	40+
	Medium/average	6-8	80% (+/-5)	35+
	Small/thin	7-9	80% (+/-5)	30+

\*Program 10 can feel quite intense and therefore the individual intensity may need adjusting for client comfort. Start lower and go up.

Remember to document your work, as these settings will vary among individuals.

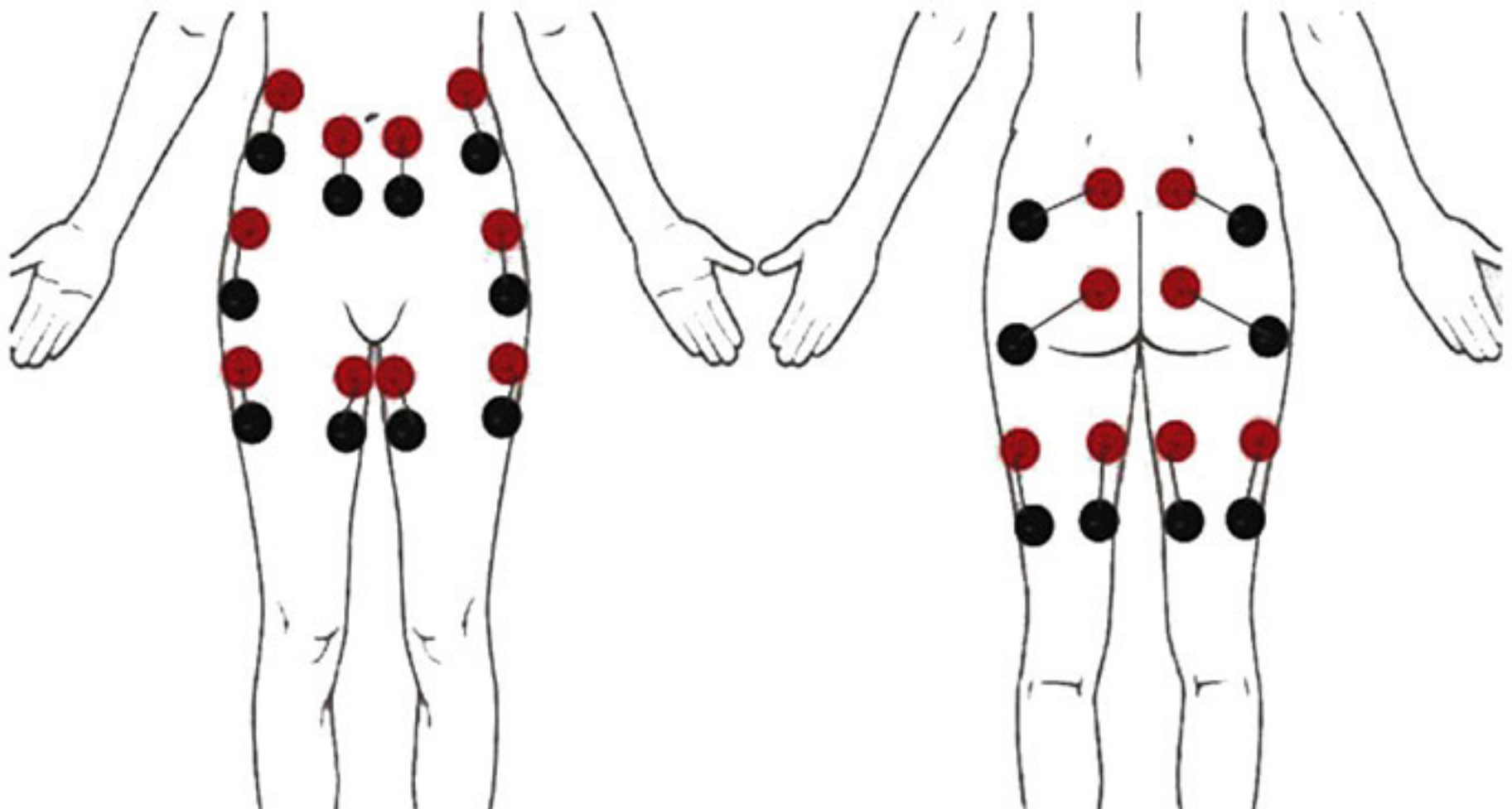

# NUE FUSION 600 MICROCURRENT

## Body Reshaping with Fat Reduction - Pad Placement


### Waist and Abdomen fat reduction & shaping

**Recommended treatments:** 2x a week for 4-6 weeks or until desired results are achieved with maintenance of 2x a month.

	Body Type	Program	Overall Intensity	Individual Intensity
<b>Phase 1:</b> fat reduction 5 minutes	All	10*	75%	25-30
<b>Phase 2:</b> reshape/tone 20-30 minutes	Large/fleshy	2-3	75% (+5)	50+
	Med/average	4-6	80% (+/-5)	45+
	Thin/toning	7-8	80% (+/-5)	40+

\*Program 10 can feel quite intense and therefore the individual intensity may need adjusting for client comfort. Start lower and go up.

Remember to document your work, as these settings will vary among individuals.

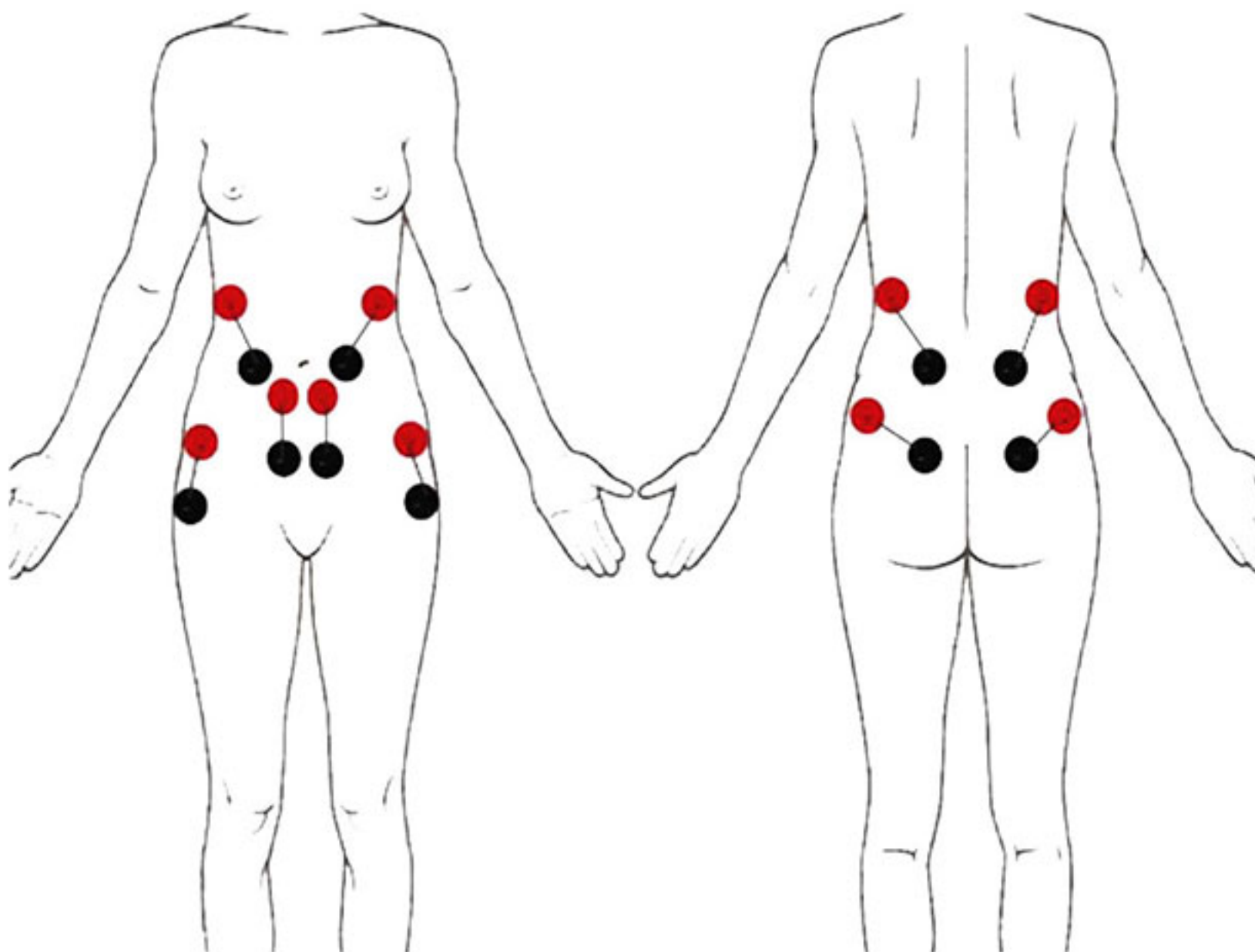

# NUE FUSION 600 MICROCURRENT

## Body Fat Reduction - Pad Placement


### Abdomen, Stomach, and Buttocks

**Recommended treatments:** 2x a week for 4-6 weeks or until desired results are achieved with maintenance of 2x a month.

	Body Type	Program	Overall Intensity	Individual Intensity
<b>Phase 1: fat reduction</b> 15-20 minutes	All	10*	75-80%	25-30
<b>Phase 2: toning</b> 10 minutes	Large/fleshy	5-6	80% (+/-5)	45+
	Med/average	7-8	80% (+/-5)	40+
	Thin/toning	8-9	80% (+/-5)	35+

\*Program 10 can feel quite intense and therefore the individual intensity may need adjusting for client comfort. Start lower and go up.

Remember to document your work, as these settings will vary among individuals.

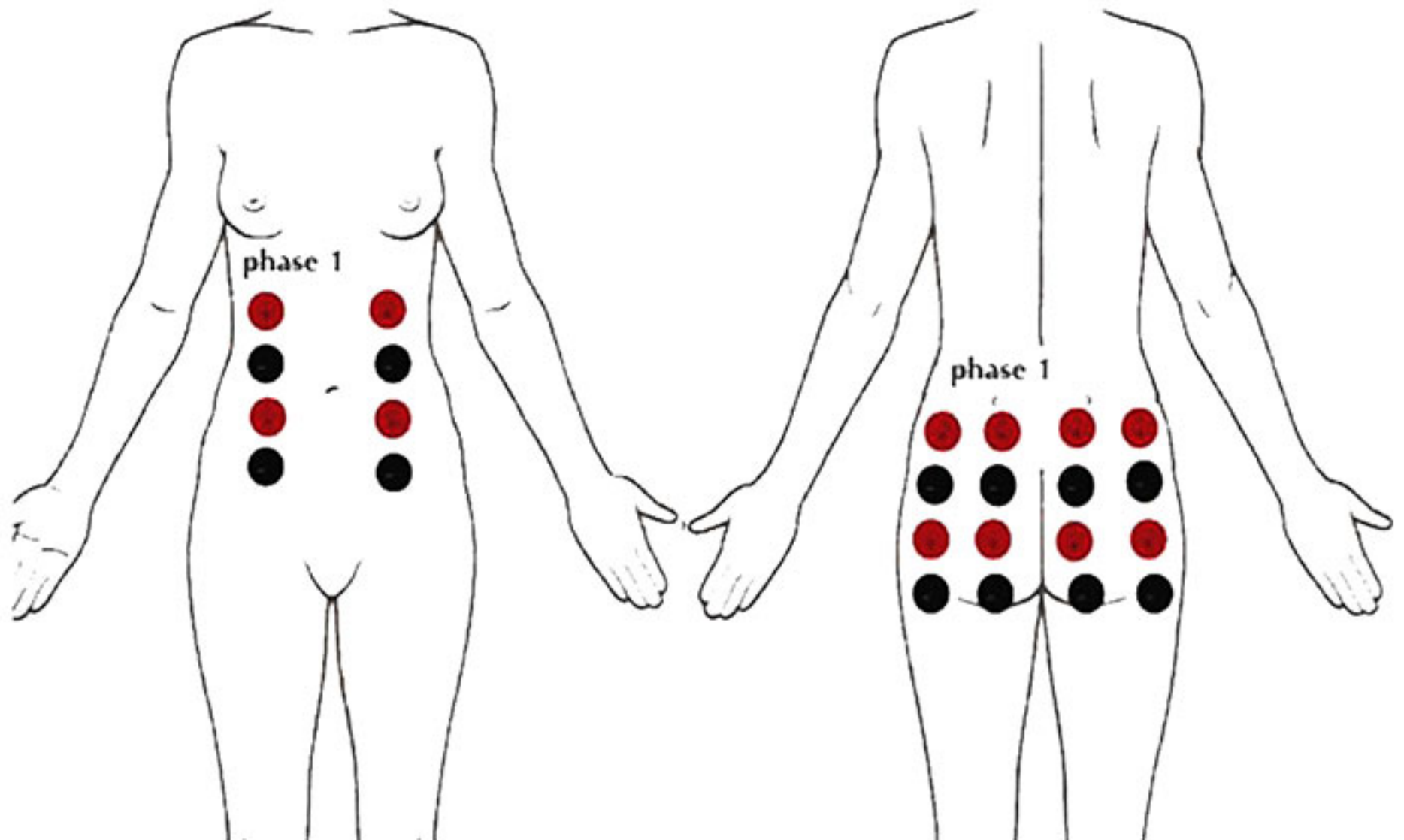

# NUE FUSION 600 MICROCURRENT

## Body Fat Reduction and Toning - Pad Placement


### Hip and Thigh

**Recommended treatments:** 2x a week for 4-6 weeks or until desired results are achieved with maintenance of 2x a month.

	Body Type	Program	Overall Intensity	Individual Intensity
Phase 1: fat reduction 15-20 minutes	All	10*	75-80%	20+
Phase 2: toning 10 minutes	Large/fleshy	4-5	75-85%	45+
	Med/average	6-7	75-85%	40+
	Thin/toning	7-8	75-85%	35+

\*Program 10 can feel quite intense and therefore the individual intensity may need adjusting for client comfort. Start lower and go up.

Remember to document your work, as these settings will vary among individuals.

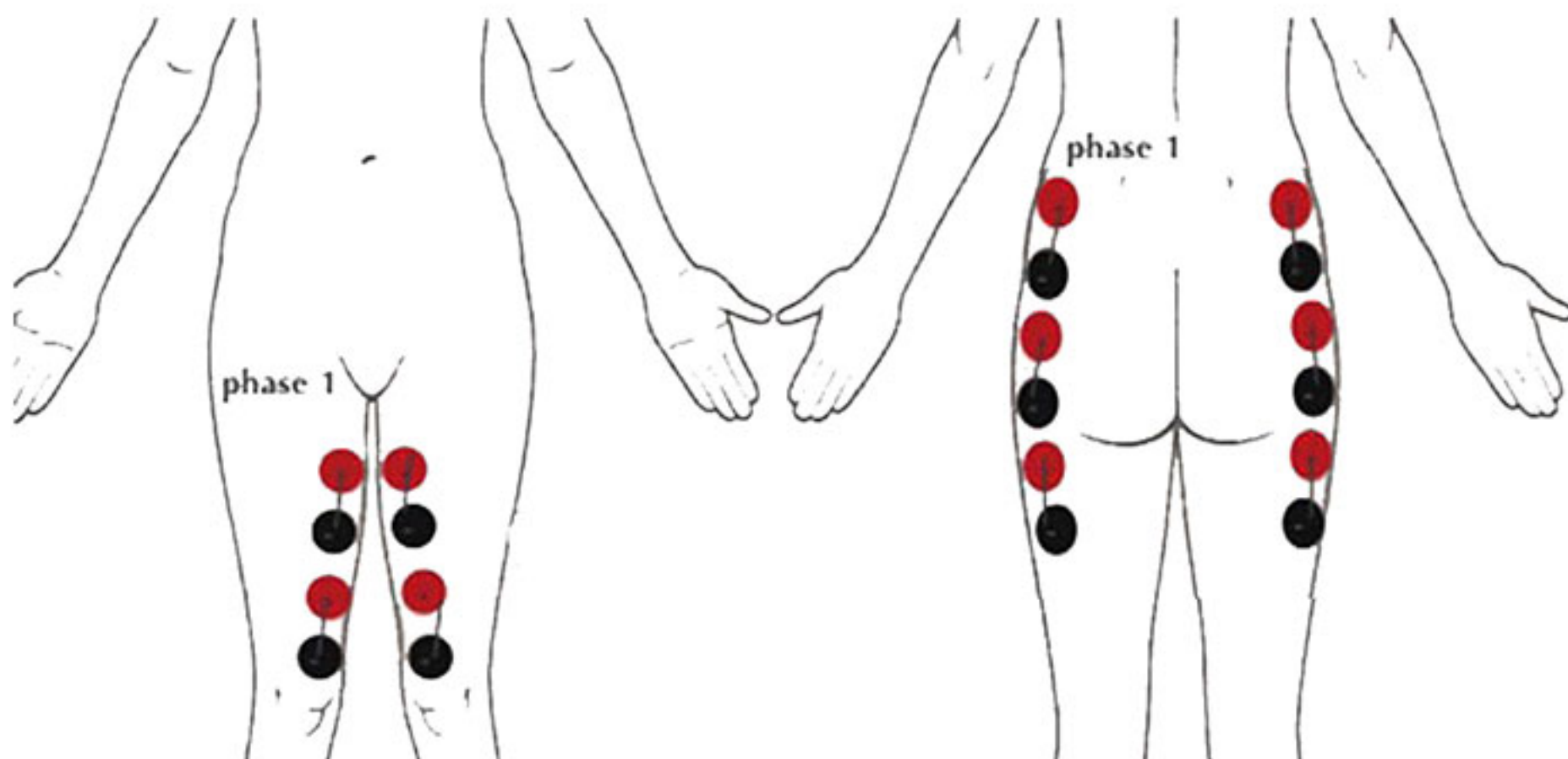



# NUE FUSION 600 MICROCURRENT

## Warranty Guidelines

### Warranty Agreement ~

1. Complete the warranty registration card and fax back to Skin for Life corporate office at (352) 797.8669 or email sales@skinforlife.com. This will enable you to take full advantage of Skin for Life's warranty coverage.
2. Skin for Life warrants this product to be free of defects in material and workmanship for a period of **two years** from the date of original purchase. (*Initial User Only - non transferable*).

### Responsibility of Original Purchaser ~

1. Return Warranty Registration Form
2. Retain Sales Receipt
3. Keep Packaging (Box and Packing Material) for at least 2 years
4. If machine requires off site service, contact Skin for Life authorized warranty service and technical center (866) 312.SKIN (7546).
5. ***If equipment is brought back to Skin for Life under warranty and it is determined that lack of maintenance is the reason why equipment is compromised, customer will incur freight and repair costs.***

### Training Support:

Equipment Training is mandatory to avoid improper use of equipment and to avoid compromising your warranty. Call Skin for Life at 866-312-7546 to schedule your initial training which may take place onsite or by web conference.

**Note\*\*\*** Maintenance is enforced along with proper training so you may maintain an excellent working machine. Without proper maintenance and training you could possibly VOID your warranty.

# NUE FUSION 600 MICROCURRENT

## Warranty Guidelines

This warranty does not cover:

1. Merchandise sold as demo units, reconditioned, floor models and/or display models.
2. Merchandise used as “Rental” equipment.
3. Merchandise that has become inoperative because of misuse, improper and/or unauthorized repair or alterations; or failure to operate product in accordance with the instructions provided in the “General Operating Manual” and reiterated in the “Mandatory” training received onsite or by webinar.
4. Repair and transportation costs if equipment is determined not to be defective.
5. Any incidental, indirect, or consequential loss, damage or expense that may result from any defect, failure or malfunction of the products.

## Skin for Life Return Policy

- All returns and/or credits are subject to approval by Skin for Life and **MUST** be made within 30 days of the original sale.
- If merchandise is returned after 30 days or without proper authorization then it is Skin for Life’s prerogative to charge a 25% restocking fee.
- Customers incur all “Shipping Costs” for the return of equipment, supplies, and/or skin care to Skin for Life.
- Everything must be returned in original “New” condition.
- Skin for Life only accepts unopened and untampered skin care products as a return. A 25% restocking fee may be applied.
- All accessories must be returned with equipment.
- Once received, Skin for Life will evaluate the equipment and accessories. Once confirmed, credit will be issued to customer. Credit Memo will be emailed to customer for confirmation of the refunded funds.

# Informed Consent ~ Sample Form

## Please Print

First Name: \_\_\_\_\_ Last Name: \_\_\_\_\_

Date of Birth: \_\_\_\_\_

Address: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

Phone: \_\_\_\_\_ Cell: \_\_\_\_\_ Work: \_\_\_\_\_

Emergency Contact: \_\_\_\_\_ Phone: \_\_\_\_\_

Physician: \_\_\_\_\_ Phone: \_\_\_\_\_

Referred By: \_\_\_\_\_

1. What is the reason for your visit today? \_\_\_\_\_

2. Are you under the care of a physician including dermatologist?  
If yes please state the reason \_\_\_\_\_

3. Are you pregnant? Y or N Are you planning to become pregnant? \_\_\_\_\_

4. Do you smoke? Y or N

5. Please circle the one that best describes your daily stress level. High, Medium, or Low Stress

6. Do you have metal in your body? Such as dental work, pins, plates, stints, etc.  
Be specific \_\_\_\_\_

7. Do you have allergies? Y or N

8. Please list ALL MEDICATIONS: \_\_\_\_\_

9. What type of exfoliation skin care products do you use? Glycolic, scrubs, retin A or any vitamin A derivatives.  
Please specify: \_\_\_\_\_

10. Do you have or have you had the following: Please check all that apply:

Epilepsy  Fever Blisters (Have you ever HAD?)  Cardiac Concerns  Acne  Keloid  
 Sinus Concerns  High Blood Pressure  Fibromyalgia  Pacemaker  Heart Stints  
 Diabetes  Botox  Kidney Disorders  Immune Disorder  Metal implants  Dental Work  
 Lupus  Mitral Valve Prolapse  Herpes  Cancer  Hyaluronic Fillers  Asthma  
 Synthetic Fillers  Plastic Surgery  Skin Cancers  Hepatitis  Herpes Zoster (Shingles)

What products are you currently using? Cleanser \_\_\_\_\_ Toner \_\_\_\_\_

Scrub \_\_\_\_\_ Creams \_\_\_\_\_ Sunscreens \_\_\_\_\_ what SPF? \_\_\_\_\_

Eye Creams \_\_\_\_\_ Serums \_\_\_\_\_

I understand the information I have given is to aid the skin therapist and is not a substitute for medical care and I understand the questions and I have answered them honestly and accurately.

Signed By Client \_\_\_\_\_ Date \_\_\_\_\_

# NUE FUSION 600 MICROCURRENT

## Suggested Informed Consent ~ Sample Form

Nue Fusion 600 Microcurrent delivers low level frequencies (*less than one millionth of an amp*) that work in harmony with the body's electrical system. Microcurrent provides skin with very small amounts of electrical stimulation, which causes a flex and relax response. This is a gentle non – invasive response to improve skin appearance.

Ideal results:

**Face Treatment** - schedule twice a week for three weeks which equals (6) skin treatments. (12) treatments are considered maximum before scheduling maintenance.

**Body Treatment** – schedule twice a week for 4 – 6 weeks which equals (8 – 12) treatments. (12) treatments are considered maximum before scheduling maintenance.

**Maintenance Treatments** – Depends on individual goals. Once or twice a month is recommended to sustain cumulative results.

**It is recommended to drink water following face or body microcurrent treatments.**

**Microcurrent Contra indications:** (*Please circle if you have any of the following*)

- Epilepsy
- Pace maker
- Melanoma
- Cancer
- Pregnancy (or if you are trying to become pregnant)
- Varicose Veins
- Phlebitis
- Herpes Zoster – Shingles

**Precautions to Consider:** (*Please circle if you have any of the following*)

- Skin Hypersensitivity
- Acne
- Rosacea
- Metal Plates or Pins
- Loss of Skin Sensation
- Stroke
- Bell's palsy
- Diabetes

I acknowledge that no guarantee has been given to me regarding how much firming and toning will result with each treatment or my final treatment. Each individual skin condition is unique and may respond differently. I understand that no specific results are guaranteed.

By signing below, I acknowledge that I have read the above information and thereby consent and agree to receive the microcurrent treatment(s). All my questions have been acknowledged and answered to my satisfaction. I hereby acknowledge that I have provided accurate and truthful information.

Client/Patient Signature: \_\_\_\_\_ Date: \_\_\_\_\_  
Witness Signature: \_\_\_\_\_ Date: \_\_\_\_\_

# L.E.D. Light Therapy

## Suggested Informed Consent ~ Sample Form

The purpose of the LED Light Therapy study from NASA was to discover the properties of wound healing. Increasing cellular growth encourages cell division which provides skin with a youthful healthy appearance. LED Light Therapy also assists with minimizing overactive oily and/or troubled skin conditions.

Ideal results – schedule twice a week for three weeks which equals (6) skin treatments.

Maintenance Treatments – Depends on individual goals. Once a week, twice a month, or once a month is recommended to sustain cumulative results.

### L.E.D. Light Therapy Contra indications:

- Epilepsy
- Do Not Apply to an Open Wound
- Thyroid Conditions
- Pregnancy (or if you are trying to become pregnant)

### Precautions to Consider:

- Sensitivity to Migraines/Headaches
- Topical or Synthetic Steroids (NSAID's)
- Do Not Use heavy creams and/or ointments

LED Light Therapy treatments have been explained to me and any questions I have regarding LED Light Therapy treatments have been answered.

I understand there are certain contraindications that preclude someone from receiving LED Light Therapy treatments and these include epilepsy, pregnancy, thyroid conditions, medications, causing light sensitivity and open wounds.

I also understand there are other precautions that should be considered before receiving LED Light treatments and may require a doctor's release and/or I assume any risk involved. These include but are not limited to a person receiving cortisone and/or steroids injections, and a person prone to light sensitive migraines.

Although LED Light Therapy is a non – invasive treatment and the precautions and contra indications have been explained to me, I understand that certain risks or complications can occur from both known and unknown causes, and I freely assume these risks.

I acknowledge that no guarantee has been given to me as to the condition of the complexion, skin pore size, fine lines, and wrinkles as results will vary per client/patient. I am aware that follow up treatments may be necessary to achieve optimal results with gradual results occurring over time.

I release \_\_\_\_\_, staff and specific technician from any liability associated with this procedure. I certify that I am of at least 18 years old or I have parental consent as indicated as co – signature below.

Client/Patient Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Co – Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Skin Professional: \_\_\_\_\_ Date: \_\_\_\_\_

# Skin for Life pure science professional skin care with Nue Fusion 600 Professional Microcurrent

When Skin for Life started manufacturing machines, we knew as a company that skin care should be a rebuilding process from progressive treatments like microcurrent.

We started working with cosmetic experts to formulate our immunity professional skin care. Our Skin for Life pure science skin care repairs, recovers, and restores all skin types, conditions, and colors without inflammation and long term irritation.

We wanted building blocks for long term healthy skin results. Our product line removes the stress, so all skin may proceed with a normal healthy function. Skin for Life trademarked LSS™ delivery “Life Sustaining System” enriches our formulas by providing key amino acids and peptides that are immunity driven for your skin’s protection and well being. Our ingredients are the innovative performance that makes Skin for Life pure science perfect for every skin condition, types, and color.

It is our pleasure to introduce you to Skin for Life pure science. You may add our signature treatments to your menu, and introduce healthy results to your clients/patients.

We look forward to hearing from you...

Enjoy, Experience, and Explore  
**Skin for Life** pure science professional skin care.

It is a Beautiful Thing to have Healthy **Skin for Life!**

# NUE FUSION 600 MICROCURRENT

## Cost per treatment... Things you need to know.

### **Nue Fusion 600 Microcurrent Face and Body:**

8oz conductivity gel – (1) ounce = \$1.75 per client/patient

5L conductivity gel – (1) ounce = \$0.18 per client/patient

### **Facial Pads:**

30R Microcurrent Pads - \$0.05 cents per treatment for (15) treatment uses

- 10 face pads for complete microcurrent facial treatment  
@ \$0.05 = \$.50 cents per treatment.

48S Microcurrent Pads - \$0.05 cents per treatment for (15) treatment uses.

- 4 face pads for neck area – for clients that have a longer neck area and can support the use of the 48S pads. 4 face pads per microcurrent face treatment @ \$0.05 = \$.20 cents per treatment.

### **Body Pads:**

48S Microcurrent Pads - you may use the following per area:

*(This is an approximate per body type).*

- Abdomen – 8 pads @ \$0.05 = \$.40 cents per treatment – normal series is twice a week for 5 weeks (10 treatments) = \$4.00 spa cost per series.
- Hips – 4 pads @ \$0.05 = \$.20 cents per treatment - normal series is twice a week for 5 weeks (10 treatments) = \$2.00 spa cost per series.
- Inner and Outer Thighs - 8 pads @ \$0.05 = \$.40 cents per treatment – normal series is twice a week for 5 weeks (10 treatments) = \$4.00 spa cost per series.

75R Microcurrent Pads – you may use the following per area:

*(This is an approximate per body type).*

- Buttocks – 8 pads @ \$0.09 = \$.72 cents per treatment - normal series is twice a week for 5 weeks (10 treatments) = \$7.20 spa cost per series.
- Posterior Thigh (ham strings) – 8 pads @ \$0.09 = \$.72 cents per treatment - normal series is twice a week for 5 weeks (10 treatments) = \$7.20 spa cost per series.

# NUE FUSION 600 MICROCURRENT Warranty Card

1. Upon receiving you order please unwrap the package and inspect your merchandise.
2. Notify Skin for Life immediately at (866) 312.SKIN (7546) if there is any damage.
3. Fill out the Warranty Card, below, and fax to (352) 797.8669.

## SKIN FOR LIFE PRODUCT WARRANTY CARD

Purchase date:	Invoice #
Model name:	Serial #
Name:	
Company Name:	
Contact Person:	
Address:	
Phone:	
Email:	
<b><i>Disclaimer: I have read the warranty provisions, disclosures and disclaimers set forth in the manual and understand said warranty is conditional upon proper maintenance and proper usage of the equipment. (signature)</i></b> _____	

You may submit by Fax: (352) 797.8669, Email: [sales@skinforlife.com](mailto:sales@skinforlife.com)  
or mail to: 15241 Telcom Dr., Brooksville, Florida 34604



# NUE FUSION 600 MICROCURRENT

## Customer Satisfaction Survey

Please take a moment to complete our customer satisfaction questionnaire and return it with your warranty card. Your opinion matters to us as we are continuously striving to improve our service to you.

1. How did you initially hear about Skin for Life?

Spa / Salon                       Esthetician                       Dermatologist  
 Cosmetologist                       Magazine                       Internet  
 Facebook / Social Media                       Trade Show  
 Referred by: \_\_\_\_\_

2. Have you purchased from Skin for Life before? Yes \_\_\_\_\_ No \_\_\_\_\_

What product(s)? \_\_\_\_\_

3. How satisfied are you with your Skin for Life experience regarding:

	Very Unsatisfied	Unsatisfied	Neutral	Satisfied	Very Satisfied
Purchase experience (placing order, delivery)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Installation / Setup (easy to follow instructions)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Initial Training (timely, thorough, easy to follow..)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Customer Service experience (knowledgeable, courteous, etc.)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Website, Social Media (ease of use, informative content, etc)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

4. Consider if you would the total package i.e. Skin for Life customer service, Nue Fusion 600 features and benefits, and cost. How satisfied are you with the Total Skin for Life Experience?

Very Unsatisfied       Unsatisfied       Neutral       Satisfied       Very Satisfied

5. If you were less than totally satisfied, what could have been done to serve you better?

---



---

Email your completed warranty card and survey to us at

[sales@skinforlife.com](mailto:sales@skinforlife.com) and **receive 10% off your next supply order.**

Thank you for your time and we appreciate your business.



Mail to: 15241 Telcom Drive  
Brooksville, FL 34604

Call toll-free US: (866) 312.SKIN (7546)

Email: [sales@skinforlife.com](mailto:sales@skinforlife.com)  
[service@skinforlife.com](mailto:service@skinforlife.com)

Visit our Websites: [www.skinforlife.com](http://www.skinforlife.com)  
[www.skinforlifepurescience.com](http://www.skinforlifepurescience.com)



You Tube



NO ANIMAL  
TESTING