



Nue Skin Oxygen

Where Nature Meets Science...



© Copyright 2014 all rights reserved
Skin for Life a division of Lifeline Medical, Inc.

ver2015-1

INTRODUCTION

OUR PROMISE TO YOU!

We are dedicated to providing reliable, affordable, and results-oriented professional equipment. Since 2003, we have been providing superior equipment, such as: microdermabrasion, microcurrent, oxygen infusion, LED Light Therapy along with performance professional skin care.

Our parent company Lifeline Medical, Inc. has been in business since 1991. A strong commitment to customer service has contributed to our success. I, as owner and President, pledge to continue this tradition and to make Skin for Life the premier customer service oriented company in this industry.

Our dedicated employees understand the importance of customer focus and we treat all our customers with integrity and respect. We realize that our ability to grow as a company is directly related to customer satisfaction and that you make the final choice when selecting a vendor.

We look forward to the opportunity to support your business with quality equipment, supplies, and skin care. Thank you for giving us a chance to earn your trust.



Tushar Shah
President and Founder of Skin for Life

Please visit us online at:
www.skinforlife.com and www.skinforlifepurescience.com

“Your All in One Solution for Equipment and Professional Skin Care”

Skin for Life's Nue Skin Oxygen is a unique delivery system using advanced product technology. Our professional treatment skin care products are the main focus for skin change. We use chemistry to promote an assimilation of a pure oxygenated environment, which assists skin cells to thrive and proliferate for a healthier life.

We use 21% oxygen (the same as we breathe) and a psi between 10 - 15 to disburse a consistent air flow that sprays a mist of our Oxygen Infusion + activator phase 2. The activator interacts with the natural enzymes within the stratum corneum which become the catalyst for breaking down the Oxygen Infusion + activator phase 2 into pure oxygen for each skin type, condition, and color.

It is the most effective way to interact with the skin and have the skin cells become a part of the process for longer lasting results. Your clients will be amazed with the results.

Skin for Life pure science trademarked LSS™ delivery, is our special system consisting of immunity amino acids and peptides. We formulated our skin care to repair, recover, and restore the skin after progressive and aggressive treatments. Our success is rebuilding the skin back to a healthier, safe and effective environment.

Our four Oxygen Infusion Gels are antibacterial, sensitive, brightening, and A, C, E complex

Oxygen Infusion Gel phase 1 + antibacterial - Has a patented ingredient “Lysozyme” that minimizes daily irritants from environmental pollutants. This assists with managing oily and troubled skin.

Oxygen Infusion Gel phase 1 + sensitive - Is the perfect solution for temporary and chronic sensitivities. Our active ingredients, chamomile and bisabolol, are natural desensitizers for irritated skin.

Oxygen Infusion Gel phase 1 + brightening - Assist in brightening and minimizing discoloration from sun damaged skin and various hormonal induced pigmentation. Active ingredients of Kojic acid, Bearberry extract, and Licorice, are brighteners that also assist with surface irritations.

Oxygen Infusion Gel phase 1 + A, C, E complex - Excellent for mature and aging skins. This invigorating performance gel creates a youthful and vibrant appearance. The triple antioxidant of A, C, E assists with the skin's natural daily protection.

Oxygen Infusion + activator phase 2 - Magical Results! Phase 2 interacts with the skin enzymes to create a more viable environment for oxygen infusion, as well as lymph circulation for healthy long term sustainable results.

Nue Skin Oxygen Infusion

Important Safety Instructions	page 5
Instructions for Use	page 6
Airbrush Maintenance	page 6
Warranty Guidelines	page 7
Warranty for Airbrush Accessory	page 8
Return Policies	page 8
How Does Oxygen Infusion Work	page 9
How Many Treatments are Needed	page 9
Who Benefits	page 9
Skin Awareness + Contra Indications	page 10
Precautions	page 10
Informed Consent ~ Sample Form	page 11
Spa Informed Consent ~ Sample Form	page 12
LED Light Summary	page 13
LED Informed Consent ~ Sample Form	page 14
(4) Oxygen Skin Therapy Protocols	
Oily / Troubled Skin	page 15
Sensitive Skin	page 16
Mature / Ageing Skin	page 17
Brightening and Lifting (for pigmentation)	page 18
Treatment Cost + Menu Pricing + Replacement Cost	page 19
Warranty Card	page 20
Customer Satisfaction Survey	page 21

Please complete and return your Warranty Card to Skin for Life through any of the following methods:

Fax: (352) 797.8669

Email: sales@skinforlife.com

Mail to: Skin for Life
15241 Telcom Dr.
Brooksville, FL 34604



Nue Skin Oxygen Infusion

Important Safety Instructions

Operation Safety Instructions –

- Read all instructions before use to avoid injury.
- To avoid serious injury or death, follow safety information in this document.
- Keep this manual for reference.
- Read all product safety labels.
- Refer to local building codes for installation requirements.
- Always turn off machine when not in use.

Safety Information:

All Electrical Devices for Prevention of Electrical Shock Exposure -

- Keep machine unplugged when setting up machine to prevent accidental injury.
- Make sure machine is on a clean and dry surface area to prevent any accidental injury.
- Prohibit water from seeping into the machine housing.
- Avoid any water and/or liquids around the vicinity of the machine when in use or not in use to prevent any accidental injury.
- Check the cord and plug for damage before use.
- Pay attention to safety labels on machine before plugging the cord into an outlet.
- Switch on the equipment only after plugging into an outlet. Always turn off the machine before unplugging the cord from the outlet.
- Allow sufficient space for ventilation to prevent overheating.
- Avoid placing the cord near any hot objects.

Nue Skin Oxygen Infusion

Instructions for Use

- 1. Installation** - Insert black hose in the front black fitting below the silver knob.
- 2. Removing the Black Hose** - Press on the black collar surrounding the black hose with your index finger and thumb. This will open the fitting to release the hose. While pressing in with your index finger and thumb, use your other hand to pull the hose free from the black fitting.
- 3. Inserting Your Power Cord** - Take your black power cord and insert in the back of the machine. There are 3 prongs, insert the power cord where they meet correctly. Press in with a firm pressure to secure the power cord to the back of the machine.
- 4. Turning On Your Equipment** -
 - a. The main power module is on the back of the machine above the black power cord. Make sure the "1" is pushed in.
 - b. The "ON" switch is on the back of the machine. You will recognize it as a silver push button switch.
- 5. Pressure Gauge - 10 psi**
 - a. Make sure the black line on the silver knob is facing the 3 O'clock position.
 - b. You may adjust the pressure depending on the area you are working on. Example: When working around the eyes you may want to adjust your setting between 5-7 psi.
 - c. Any areas you may want to isolate, please adjust silver knob for comfortable air flow.

If you have any concerns or questions with installation please call Skin for Life at 866.312.7546 or email service@skinforlife.com.

Airbrush Maintenance

To Avoid Compromising Your Warranty with Skin for Life, the following maintenance is Mandatory:

- 1. After every treatment, use Skin for Life O² cleaner.**
 - a. After your treatment, fill your airbrush cup 1/4 full with the O² cleaner, and spray directly into gauze/towel letting the tip of the gun touch the gauze or towel. This will allow the solution to dislodge any debris around the needle of the airbrush.
 - b. Once you have completed the O² cleaning step, fill the cup 1/4 full with water, and spray until empty.
 - c. This will complete your cleaning process and you may begin your next oxygen infusion treatment.
 - d. This is a 30 second - 1 minute process and should be considered a part of your routine clean up after your skin therapy.



Nue Skin Oxygen Infusion Equipment

Warranty Guidelines

Warranty Agreement ~

1. Complete the warranty registration card and fax back to Skin for Life corporate office at (352) 797.8669 or email sales@skinforlife.com. This will enable you to take full advantage of Skin for Life's warranty coverage.
2. Skin for Life warrants this product to be free of defects in material and workmanship for a period of **two years** from the date of original purchase. (*Initial User Only - non transferable*).

Responsibility of Original Purchaser ~

1. Return Warranty Registration Form
2. Retain Sales Receipt
3. Keep Packaging (Box and Packing Material) for at least 2 years
4. If machine requires off site service, contact Skin for Life authorized warranty service and technical center (866) 312.SKIN (7546).
5. ***If equipment is brought back to Skin for Life under warranty and it is determined that lack of maintenance is the reason why equipment is compromised, customer will incur freight and repair costs.***

This Warranty does not cover:

1. Merchandise sold as demo units, reconditioned, floor models and/or display models.
2. Merchandise used as "Rental" equipment.
3. Merchandise that has become inoperative because of misuse, improper and/or unauthorized repair or alterations; or failure to operate product in accordance with the instructions provided in the "General Operating Manual" and reiterated in the "Mandatory" training received onsite or by webinar.
4. Repair and transportation costs if equipment is determined not to be defective.
5. Any incidental, indirect, or consequential loss, damage or expense that may result from any defect, failure or malfunction of the products.

Training Support:

Equipment Training is mandatory to avoid improper use of equipment and to avoid compromising your warranty. Call Skin for Life at (866) 312.7546 to schedule your initial training which may take place onsite or by web conference.

Note*** Maintenance is enforced along with proper training so you may maintain an excellent working machine. Without proper maintenance and training you could possibly VOID your warranty.

Nue Skin Oxygen Infusion Equipment

Warranty Guidelines

Warranty for Airbrush Accessory

All Skin for Life accessories have a 90 day warranty.

If within 90 days you have a concern with your airbrush accessory, we will replace your airbrush gun under your warranty agreement. However you must comply with the following steps below in order for us to ship your warranty replacement.

1. Please call Skin for Life (866).312.SKIN (7546) and a customer service representative will give you a RMA# so you may write it on your package for a quick Skin for Life response once we receive your package.
2. Mail your defective airbrush gun through USPS (postal service).
3. Once your airbrush is evaluated, if it is not due to a maintenance error, and it is clearly a defective gun we will refund your postal charge and send you a warranty replacement as quickly as possible.

Skin for Life's recommendation is to purchase a separate Airbrush Gun as a reserve, so you are never without the proper tools to complete your skin therapy.

Skin for Life Return Policy:

- All returns and/or credits are subject to approval by Skin for Life and MUST be made within 30 days of the original sale.
- If merchandise is returned after 30 days or without proper authorization then it is Skin for Life's prerogative to charge a 25% restocking fee.
- Customers incur all "Shipping Costs" for the return of equipment, supplies, and/or skin care to Skin for Life.
- Everything must be returned in original "New" condition.
- Skin for Life only accepts unopened and untampered skin care product as a return. A 25% restocking fee may be applied.
- All accessories must be returned with equipment.
- Once received, Skin for Life will evaluate the equipment and accessories. Once confirmed, credit will be issued to customer. Credit Memo will be emailed to customer for confirmation of the refunded funds.

How Does Nue Skin Oxygen Work?

The Nue Skin Oxygen system has an oxygen motor and pump to dispense 21% oxygen onto the skin. To begin the treatment you will need to use one of the four Oxygen Infusion Gels phase 1, apply with a facial mask brush for a medium thick application. Once you have completed your application, you may use your performance massage movements for 1 - 2 minutes. Fill your airbrush cup with Oxygen Infusion phase 2 + activator. Adjust your pressure to a comfortable emission level of oxygen. Apply the nozzle of the airbrush close to the skin, where the airbrush nozzle is almost touching the skin. Use small circular movements with your airbrush gun and then use the other hand to manipulate the Oxygen Infusion phase 2 + activator until you see a white lather to the skin. This will signify interaction between the Oxygen Infusion phase 2 + activator and stratum corneum to assist the skin's natural process of breaking down our formulation into your skin's pure oxygen. You may perform 2-3 additional passes to complete the application. Once you complete your passes, use your performance massage movements for 1-2 minutes and then remove with a warm towel and continue with your skin therapy steps.

How Many Treatments are Needed?

You may add Skin for Life Oxygen Infusion Therapy to your clients/patients monthly facial treatment.

You may see your client/patient twice a week for 3 weeks to meet their skin goals, (pigmentation, sensitivity, oily/troubled skin, or mature/aging skin).

You may see your client/patient every two weeks to maintain and sustain their skin goals.

Who Benefits?

Every skin type, condition, and color benefits from Skin for Life Oxygen Infusion therapies. Our formulations are performance driven, providing healthy long lasting results for a youthful appearance.

Nue Skin Oxygen Infusion

Skin Awareness + Contra Indications

- Pregnancy
- Breast Feeding
- Accutane - (Should be a minimum of 8 months), when in doubt consult physician for approval.
- Do Not combine with TCA, Jessner, or Salicylic Acid Peels
- Sunburn
- Rosacea Advanced Stages - III
- Cancer/Skin Cancers/Open Lesions

Precautions

- Alpha Hydroxy (Glycolic, Mandelic, Azelaic, Lactic, Citric) skin must be free of all sensitivities before applying Skin for Life oxygen infusion therapy.
- Retinoid (Retin A, Tazorac, Differin, and any other vitamin A derivative) Have client/patient discontinue for 3 days before performing an oxygen infusion therapy.
- Skin sensitivity
- Contacts - Please have client/patient remove contacts before oxygen procedure.
- Under the care of a physician - general for all procedures administered.

Informed Consent ~ Sample Form

Please Print

First Name: _____ Last Name: _____

Date of Birth: _____

Address: _____

City: _____ State: _____ Zip: _____

Phone: _____ Cell: _____ Work: _____

Emergency Contact: _____ Phone: _____

Physician: _____ Phone: _____

Referred By: _____

1. What is the reason for your visit today? _____

2. Are you under the care of a physician including dermatologist?
If yes please state the reason _____

3. Are you pregnant? Y or N Are you planning to become pregnant? _____

4. Do you smoke? Y or N

5. Please circle the one that best describes your daily stress level. High, Medium, or Low Stress

6. Do you have metal in your body? Such as dental work, pins, plates, stints, etc.
Be specific _____

7. Do you have allergies? Y or N

8. Please list ALL MEDICATIONS: _____

9. What type of exfoliation skin care products do you use? Glycolic, scrubs, retin A or any vitamin A derivatives.
Please specify: _____

10. Do you have or have you had the following: Please check all that apply:

Epilepsy Fever Blisters (Have you ever HAD?) Cardiac Concerns Acne Keloid
 Sinus Concerns High Blood Pressure Fibromyalgia Pacemaker Heart Stints
 Diabetes Botox Kidney Disorders Immune Disorder Metal implants Dental Work
 Lupus Mitral Valve Prolapse Herpes Cancer Hyaluronic Fillers Asthma
 Synthetic Fillers Plastic Surgery Skin Cancers Hepatitis Herpes Zoster (Shingles)

What products are you currently using? Cleanser _____ Toner _____

Scrub _____ Creams _____ Sunscreens _____ what SPF? _____

Eye Creams _____ Serums _____

I understand the information I have given is to aid the skin therapist and is not a substitute for medical care and I understand the questions and I have answered them honestly and accurately.

Signed By Client _____ Date _____

Spa Informed Consent ~ Sample Form

- Do not wear contact lenses during treatment sessions - they may be placed back in after treatment.
- Do not use glycolic, AHA's, or any retinol products 48 hours before or after treatment.
- You must not be currently taking Accutane. Nor have you taken Accutane for the previous 8 months to 1 year.
- No suntan beds for 2 weeks post treatment (recommend self tanners for a healthy lifestyle change).
- Physical sunscreen of at least SPF 30 to be applied and re applied during sun exposure.

Initials Required

_____ I have been thoroughly informed by professional therapist _____ of any complications to recommended procedure _____ and I understand there are no guarantees to procedures given by _____.

_____ I am committed to following post treatment care given to me by professional therapist _____ to ensure the best possible result.

_____ I have given all medical information to the best of my knowledge to _____ to prevent any contra indications to procedures.

Yes or No - I have had a history of cold sores, fever blisters, and/or shingles.

Yes or No - I am using (Retin A, Tretinoic Acid, Differin, Glycolic Acid, Accutane, or other vitamin A derivative topical creams and/or ointments.

_____ I have not used any performance agents within the last 5 - 7 days, knowing that this is a contra indicator for having procedures performed.

_____ I am committed to avoiding the sun and/or tanning facilities for the recommended time suggested by professional therapist _____.

Client Signature: _____ Date: _____

Professional Therapist Signature: _____ Date: _____


LED Light Summary

LED technology was originally developed to enhance the growth of plant tissue in space by NASA's Marshall Space Flight Center and Quantum Devices, Inc. of Barneveld, Wisconsin. LED's have a similar physiological effect on human cells as they do on plant cells. It is a natural photo – biochemical reaction that is similar to photosynthesis in plants, which triggers the body to convert light energy into cell energy.

**Age defying - 680nm
(RED)**

**Oily/Troubled Skin - 460nm
(BLUE)**

**Skin Rebuilding - 800nm
(Infrared)**

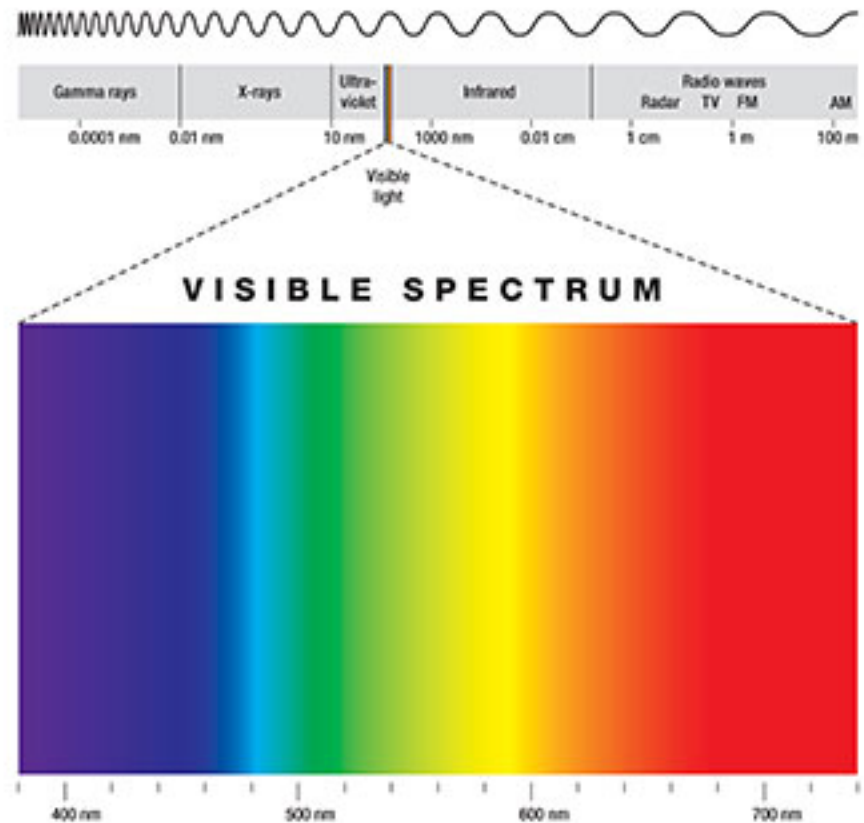

This remarkable technology, enhances the ATP (Adenosine Tri phosphate) within the mitochondria of the cells which excites the first process of cell division. This non thermal response relies on the visible light spectrum to interact with cells for a growth response.

RED LED Light is designed to trigger the first initial ATP response. RED LED Light should always be the first light used on all skin types, conditions, and colors.

Skin Indications – sun damaged skin, mature, pigmented lesions, oily, troubled skin, capillary distention, and temporary/chronic sensitive skins.

How To Use – RED hand piece – use 90 seconds per area, with overlapping 30% - 50% completing the process within 12 – 15 minutes before transitioning to another LED Light color.

RED LED Light Stand Alone Standard Head Unit – use 20 minutes before transitioning to another LED Light color.

BLUE LED Light is suggested for oily and troubled skin conditions. The depth of this light is important to minimize the activity of bacteria within the sebaceous gland that is associated with blemished skins.

Skin Indications – oily, troubled skin, blemished skin, and seborrhea skins.

How To Use – BLUE LED Light hand piece for 90 seconds per area with overlapping 30% - 50% completing the process in 10 minutes.

BLUE LED Light Stand Alone Unit – use 10 – 15 minutes after the RED LED Light stand-alone unit.

GREEN LED Light is recommended for pigmented lesions, like, melasma, chloasma, and solar lentigines. **Skin Indications** – discolored skin, dark spots, age spots, pregnancy mask, melasma. **How To Use** – use 10 – 15 minutes after the RED LED Light Stand Alone unit. *(NO hand pieces available)*

YELLOW LED Light assist lymphatic drainage, with edema and erythema conditions. **Skin Indications** – swelling, edema, redness, erythema **How To Use** – use 10 – 15 minutes after RED LED Light Stand Alone unit. *(NO hand pieces available)*

L.E.D. Light Therapy

Suggested Informed Consent ~ Sample Form

The purpose of the LED Light Therapy study from NASA was to discover the properties of wound healing. Increasing cellular growth encourages cell division which provides skin with a youthful healthy appearance. LED Light Therapy also assists with minimizing overactive oily and/or troubled skin conditions.

Ideal results – schedule twice a week for three weeks which equals (6) skin treatments.

Maintenance Treatments – Depends on individual goals. Once a week, twice a month, or once a month is recommended to sustain cumulative results.

L.E.D. Light Therapy Contra indications:

- Epilepsy
- Do Not Apply to an Open Wound
- Thyroid Conditions
- Pregnancy (or if you are trying to become pregnant)

Precautions to Consider:

- Sensitivity to Migraines/Headaches
- Topical or Synthetic Steroids (NSAID's)
- Do Not Use heavy creams and/or ointments

LED Light Therapy treatments have been explained to me and any questions I have regarding LED Light Therapy treatments have been answered.

I understand there are certain contraindications that preclude someone from receiving LED Light Therapy treatments and these include epilepsy, pregnancy, thyroid conditions, medications, causing light sensitivity and open wounds.

I also understand there are other precautions that should be considered before receiving LED Light treatments and may require a doctor's release and/or I assume any risk involved. These include but are not limited to a person receiving cortisone and/or steroids injections, and a person prone to light sensitive migraines.

Although LED Light Therapy is a non – invasive treatment and the precautions and contra indications have been explained to me, I understand that certain risks or complications can occur from both known and unknown causes, and I freely assume these risks.

I acknowledge that no guarantee has been given to me as to the condition of the complexion, skin pore size, fine lines, and wrinkles as results will vary per client/patient. I am aware that follow up treatments may be necessary to achieve optimal results with gradual results occurring over time.

I release _____, staff and specific technician from any liability associated with this procedure. I certify that I am of at least 18 years old or I have parental consent as indicated as co – signature below.

Client/Patient Signature: _____ Date: _____

Co – Signature: _____ Date: _____

Skin Professional: _____ Date: _____

Nue Skin Oxygen Infusion Protocol

Oily / Troubled Skin

- Cleanse:** Salicylic Gel Cleanser + clarifying solution - put (1) teaspoon of Salicylic Gel Cleanser in a glass bowl and whip to a creamy lather and apply to face and neck. Manipulate with gentle massage movements and remove with a warm towel.
- Tone:** Use Salicylic Toner + eucalyptus oil - apply to the entire face and neck.
- Exfoliation**, may use the following:
 - Microdermabrasion (Non-inflamed skin only)
 - Enzyme Exfoliate + papaya complex
 - Dermaplaning (Non-inflamed skin only)
 - Alpha Hydroxy Acids (however make sure there are NO sensitivities before applying Skin for Life oxygen products).
 - Ultra Scrubber - cavitations
- Oxygen Infusion Gel phase 1 + antibacterial with LSS™ delivery** - Apply a medium thickness with a facial mask brush to the entire face and neck. You may concentrate on localized areas of concern. Manipulate for 1-2 minutes with performance massage movements.
- Oxygen Infusion phase 2 + activator** - Fill the cup of your airbrush gun, and spray onto the skin in circular movements. The tip of the airbrush gun should be very close to the skin for liquid concentration. You may manipulate with the other hand as you spray the oxygen infusion phase 2 + activator. You may perform 2-3 passes to complete the process. Once completed use performance massage movements for 1-2 minutes and then remove with warm towel.
- LED Light Therapy**, may use the following: (Optional)
 - RED L.E.D. Light Hand Piece - Apply to the skin for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another area, again by holding to the area for 90 seconds. Perform 4-5 passes before using the BLUE L.E.D. Light hand piece.
 - For non-inflamed and/or inflamed troubled skin, you may use BLUE L.E.D. Light hand piece. Repeat the same movements from your RED L.E.D. Light application .
- Active + Restore** -
 - Apply Moisture Booster + hyaluronic acid with LSS™ delivery and Vita C Peptide Serum with LSS™ delivery.
- Nourish + Protect** -
 - Apply Antibac Essentials + lysozyme protection with LSS™ delivery and Vital Recovery Concentrate + natural botanicals with LSS™ delivery.
- Physical Protection** -
 - SOL defense SPF 30 Broad Spectrum UVA & UVB

You may also apply "Algae 2 part Mask + Activator" after Nourish + Protect to cool, calm, and refresh the skin, wait 5-7 minutes then remove the mask and repeat by finishing with serum, moisturizer, and sunscreen.

Nue Skin Oxygen Infusion Protocol

Sensitive Skin

1. **Cleanse:** Gentle Gel Cleanser + soothing aloe or Gentle Milk Cleanser + vitamins A & E - Apply a generous amount to the face, neck, and chest area. Do not add water to the cleanser, let the cleanser bind with the skin for a proper emulsion to breakdown debris and other daily pollutants. Remove with warm gauze, sponges, or towels.
2. **Tone:** Instant Soothing Toner + natural actives - Spray over the face or you may use a moist gauze, this will cool and ready the skin for the next application.
3. **Exfoliate:** Enzyme Exfoliate + papaya complex with LSS™ delivery - Squeeze a generous amount into a small glass bowl and apply the papaya enzyme with a facial mask brush to the entire face, neck, and chest with a medium thick application. You may use steam to stimulate the papaya or moisten your finger tips and start with fluid massage movements to stimulate the process of exfoliation. Remove with a warm towel.
4. **Tone:** Instant Soothing Toner + natural actives - Spray over the entire face and finish by removing any residue with moist gauze, sponges, or a towel.
5. **Oxygen Infusion Gel phase 1** + sensitive with LSS delivery - Apply a medium thickness with a facial mask brush to the entire face and neck. Manipulate for 1-2 minutes with performance massage movements. Do Not Remove.
6. **Oxygen Infusion phase 2** + activator - Fill the cup of your airbrush gun, and spray onto the skin in circular movements. The tip of the airbrush gun should be very close to the skin for liquid concentration. You may manipulate with the other hand as you spray the oxygen infusion phase 2 + activator. You may perform 2-3 passes to complete the process. Once completed use performance massage movements for 1-2 minutes and then remove with warm towel.
7. **LED Light Therapy**, may use the following: (Optional)
 - a. RED L.E.D. Light Hand Piece - Apply to the skin for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece on another area, again by holding to the area for 90 seconds. The complete application time is 12 - 15 minutes for significant results.
8. **Active + Restore:** Vita C Peptide Serum with LSS™ delivery, and Moisture Booster + hyaluronic acid with LSS™ delivery. Apply (1) pump to the face, neck, and chest.
9. **Nourish + Protect:** Vital Recovery Concentrate + natural botanicals - Micro circulating properties with calming benefits to irritated skins. This is your ZEN recovery for temporary and chronic sensitive skins.
10. **Replenish + Restore:** Mix Algae Mask part 1 with Algae activator part 2, blend until it is a smooth consistency. Apply quickly to the entire face, if applying to neck and chest use another 2 part Algae system. Wait 5-7 minutes and remove. Mask will solidify for a easy removal.
11. Apply Vita C Peptide Serum with LSS™ delivery, Vital Recovery Concentrate + natural botanicals with LSS™ delivery, and SOL defense SPF 30 Broad Spectrum sunscreen.

Nue Skin Oxygen Infusion Protocol

Mature / Ageing

1. **Cleanse:** Gentle Milk Cleanser + vitamins A & E - Apply a generous amount to the face, neck, and chest and massage to emulsify debris and other daily pollutants. Do Not use water with the cleanser, it works best alone when binding with the skin.
2. **Tone:** Instant Soothing Toner + natural actives - Spray over the entire face and neck, and then you may use moisten gauze, sponge, and/or a warm towel to remove residue. This prepares the skin for the next application.
3. **Exfoliation:** You may use one or two of the following:
 - a. Microdermabrasion
 - b. Ultra Scrubber
 - c. Alpha Hydroxy (Lactic, Glycolic, Mandelaic, Citric...) Make sure the skin has NO sensitivities before applying Oxygen Infusion Gels.
 - d. Dermaplaning
 - e. Enzyme Exfoliate + papaya complex
4. **Oxygen Infusion Gel phase 1 + A, C, E complex with LSS™ delivery** - Apply a medium thickness with a facial mask brush to the entire face and neck. Manipulate for 1-2 minutes with performance massage movements. Do Not Remove.
5. **Oxygen Infusion phase 2 + activator** - Fill the cup of your airbrush gun, and spray onto the skin in circular movements. The tip of the airbrush gun should be very close to the skin for liquid concentration. You may manipulate with the other hand as you spray the oxygen infusion phase 2 + activator. You may perform 2-3 passes to complete the process. Once completed use performance massage movements for 1-2 minutes and then remove with a warm towel.
6. **LED Light Therapy**, may use the following: (Optional)
 - a. RED L.E.D. Light Hand Piece - Apply to the skin for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another area, again by holding to the area for 90 seconds. The complete application time is 12 - 15 minutes for significant results.
7. **Active + Restore:** Vita C Peptide Serum with LSS™ delivery and Moisture Booster + hyaluronic acid with LSS™ delivery. Apply (1) pump and blend well.
8. **Nourish + Protect:** Vital Recovery Concentrate + natural botanical with LSS™ delivery and Ultra Performing Complex + A, C, E complex with LSS™ delivery. Apply (1) pump and blend well.
9. **Massage with lifting + firming movements.** This is a great time to perform Nue Fusion Microcurrent, to lift, firm, and tighten the skin and encourage muscle stimulation. Follow the microcurrent protocol at this point.
10. **Replenish + Restore:** Mix Algae Mask part 1 with Algae activator part 2, blend until it is a smooth consistency. Apply quickly to the entire face, if applying to neck and chest use another 2 part Algae system. Wait 5-7 minutes and remove. Mask will solidify for a easy removal.
11. Apply Vita C Peptide Serum with LSS™ delivery, Moisture Booster + hyaluronic acid with LSS™ delivery, Ultra Performing Complex + A, C, E complex with LSS™ delivery, and SOL defense SPF 30 Broad Spectrum sunscreen as your complete skin protection.

Nue Skin Oxygen Infusion Protocol

Brightening and Lifting

1. **Cleanse:** Gentle Milk Cleanser + vitamins A & E - Apply a generous amount to the entire face, neck, and chest, and manipulate in circular movements. Do Not add water, let the cleanser bind with the skin to emulsify debris and daily pollutants.
2. **Tone:** Instant Soothing Toner + natural actives - Spray over the entire face and neck, and then you may use moisten gauze, sponge, and or towel to remove residue. This prepares the skin for the next application.
3. **Exfoliation:** You may use one or two of the following:
 - a. Microdermabrasion
 - b. Ultra Scrubber
 - c. Alpha Hydroxy (Lactic, Glycolic, Mandelaic, Citric...) Make sure the skin has NO sensitivities before applying Oxygen Infusion Gels.
 - d. Dermaplaning
 - e. Enzyme Exfoliate + papaya complex
4. **Oxygen Infusion Gel phase 1 + brightening with LSS™ delivery** - Apply a medium thickness with a facial mask brush to the entire face and neck. Manipulate for 1-2 minutes with performance massage movements. Do Not Remove.
5. **Oxygen Infusion phase 2 + activator** - Fill the cup of your airbrush gun, and spray onto the skin in circular movements. The tip of the airbrush gun should be very close to the skin for liquid concentration. You may manipulate with the other hand as you spray the oxygen infusion phase 2 + activator. You may perform 2-3 passes to complete the process. Once completed use performance massage movements for 1-2 minutes and then remove with warm towel.
6. **LED Light Therapy**, may use the following: (Optional)
 - a. RED L.E.D. Light Hand Piece - Apply to the skin for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another area, again by holding to the area for 90 seconds. You may perform the application for 4 - 5 cycles over the entire face. The complete application time is 12 - 15 minutes for significant results.
7. **Active + Restore:** Vita C Peptide Serum with LSS™ delivery and Moisture Booster + hyaluronic acid with LSS™ delivery. Apply (1) pump and blend well.
8. **Nourish + Protect:** Vital Recovery Concentrate + natural botanical with LSS™ delivery and Ultra Performing Complex + A, C, E complex with LSS™ delivery. Apply (1) pump and blend well.
9. **Massage with lifting + firming movements.** This is a great time to perform Nue Fusion Microcurrent, to lift, firm, and tighten the skin and encourage muscle stimulation. Follow the microcurrent protocol at this point.
10. **Replenish + Restore:** Mix Algae Mask part 1 with Algae activator part 2, blend until it is a smooth consistency. Apply quickly to the entire face, if applying to neck and chest use another 2 part Algae system. Wait 5-7 minutes and remove. Mask will solidify for a easy removal.
11. Apply Vita C Peptide Serum with LSS™ delivery, Moisture Booster + hyaluronic acid with LSS™ delivery, Ultra Performing Complex + A, C, E complex with LSS™ delivery, and SOL defense SPF 30 Broad Spectrum sunscreen for your complete skin protection.

Nue Skin Oxygen Infusion

Professional Treatment Cost

Oxygen Infusion Gel phase 1 - Approximation from using 7ml - \$1.93 per treatment

Oxygen Infusion phase 2 + activator - Approximation from using 5ml - \$1.38 each

Total Treatment Cost: \$3.31 per treatment

Menu Pricing

30 minute - Oxygen Infusion Signature Treatment	\$65.00
- Your Profit - \$61.69 equates to \$2.06 per minute.	
1 hour - Oxygen Infusion Signature Treatment	\$95.00
- Your Profit - \$91.69 equates to \$3.06 per minute	
Add-on Service	\$35.00
- Your Profit - \$31.69 equates to a 5 minute add on to a skin therapy service - \$6.34 per minute.	

Replacement Cost

Oxygen Infusion Gel phase 1 (A, C, E complex, brightening, antibacterial, sensitive)	\$ 33.00 each
Oxygen Infusion phase 2 + activator	\$ 33.00
Oxygen Airbrush Accessory - Silver with Cap	\$125.00
O2 Cleaner	\$ 5.00

Nue Skin Oxygen Infusion Equipment Warranty Card

1. Upon receiving you order please unwrap the package and inspect your merchandise.
2. Notify Skin for Life immediately at (866) 312.SKIN (7546) if there is any damage.
3. Fill out the Warranty Card, below, and fax to (352) 797.8669.

SKIN FOR LIFE PRODUCT WARRANTY CARD	
Purchase date:	Invoice #
Model name:	Serial #
Name:	
Company Name:	
Contact Person:	
Address:	
Phone:	
Email:	
<p><i>Disclaimer: I have read the warranty provisions, disclosures and disclaimers set forth in the manual and understand said warranty is conditional upon proper maintenance and proper usage of the equipment. (signature)_____</i></p>	

You may submit by Fax: (352) 797.8669, Email: sales@skinforlife.com
or mail to: 15241 Telcom Dr., Brooksville, Florida 34604

Nue Skin Oxygen Infusion Equipment

Customer Satisfaction Survey

Please take a moment to complete our customer satisfaction questionnaire and return it with your warranty card. Your opinion matters to us as we are continuously striving to improve our service to you.

1. How did you initially hear about Skin for Life?

Spa / Salon Esthetician Dermatologist
 Cosmetologist Magazine Internet
 Facebook / Social Media Trade Show
 Referred by: _____

2. Have you purchased from Skin for Life before? Yes _____ No _____

What product(s)? _____

3. How satisfied are you with your Skin for Life experience regarding:

	Very Unsatisfied	Unsatisfied	Neutral	Satisfied	Very Satisfied
Purchase experience (placing order, delivery)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Installation / Setup (easy to follow instructions)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Initial Training (timely, thorough, easy to follow..)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Customer Service experience (knowledgeable, courteous, etc.)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Website, Social Media (ease of use, informative content, etc)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

4. Consider if you would the total package i.e. Skin for Life customer service, Nue Skin Oxygen features and benefits, and cost. How satisfied are you with the Total Skin for Life Experience?

Very Unsatisfied	Unsatisfied	Neutral	Satisfied	Very Satisfied
<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

5. If you were less than totally satisfied, what could have been done to serve you better?

Email your completed warranty card and survey to us at

sales@skinforlife.com and **receive 10% off your next supply order.**

Thank you for your time and we appreciate your business.



Mail to: 15241 Telcom Drive
Brooksville, FL 34604

Call toll-free US: (866) 312.SKIN (7546)

Email: sales@skinforlife.com
service@skinforlife.com

Visit our Websites: www.skinforlife.com
www.skinforlifepurescience.com



YouTube



NO ANIMAL
TESTING