

Airbrush Maintenance

To Avoid Compromising Your Warranty with Skin for Life, the following is Mandatory.

1. After every treatment, use Skin for Life O² cleaner.
 - a. After your treatment, fill your airbrush cup 1/4 full with the O² cleaner, and spray directly into gauze or towel letting the tip of the gun touch the gauze or towel. This will allow the solution to dislodge any debris around the needle of the airbrush.
 - b. Once you have completed the cleaning process, fill the cup 1/4 full with water, and spray until empty.
 - c. This will complete your cleaning process and you may begin your next oxygen infusion treatment.
 - d. This is a 30 second - 1 minute process, and should be considered a part of your clean up after your skin therapy.

Warranty for Airbrush Accessory

All Skin for Life accessories have a 90 days warranty.

If within 90 days you have a concern with your airbrush accessory, we will replace your airbrush gun under your warranty agreement. However you must comply with the following steps below in order for us to ship your warranty replacement.

1. Please call Skin for Life (866).312.SKIN (7546) and a customer service representative will give you a RMA# so you may write it on your package for a quick Skin for Life response once we receive your package.
2. Mail your defective airbrush gun through USPS (postal service).
3. Once your airbrush is evaluated, if it is not due to a maintenance error, and it is clearly a defective gun we will refund your postal charge and send you a warranty replacement as quickly as possible.

*Skin for Life's Recommendation is to purchase a separate Airbrush Gun as a reserve,
so you are never without the proper tools to complete your skin therapy.*