

Things to **MAINTAIN:**

Nue Skin 50 & Organic Microdermabrasion **THINGS** to Try Before You Call



FILTERS

Blue LT Filter -

- The blue LT Filter needs to be changed after approximately 12-15 treatments.
- If the filter has been changed and the microdermabrasion machine is still not working correctly, it might be the internal filter. Please call service for further instructions.

Foam Filter -

- The gray Foam Filter needs to be cleaned with *compressed air* after each treatment and is to be changed after 10 treatments.

Additional Trouble-shooting

- Remove both the LT Filter and Foam Filter. With the machine on, place your finger over the hole where the filter screws in. This should let you read the maximum performance of the compressor. If the machine reading is not at least 22 Hg/inches (+/- 1) then please call service for instructions.

POOR GRAIN/CRYSTAL FLOW



Sipper Tube -

- This is most likely due to a clogged Sipper Tube.

Inside the Material Canisters are Three Metallic Tubes -

- The agitator tube in front with a gold cone
- Two sipper tubes in back
- On the short side of the coupling, you will see two set screws – one has a hole and one does not.
- Using your wire dowel that came with your machine, poke the wire end inside the hole in the set screw to loosen any debris possibly clogging the hole.
- After you have done this, insert the straw of the can of compressed air into the hole, blowing air through the opening to flush any additional debris.