



**PURE O<sub>2</sub> PORTABLE**  
**oxygen infusion**  
professional machine



**MADE IN  
U.S.A.**



## MEET TUSHAR SHAH

owner & founder of Skin for Life

### Our Promise to You

We are dedicated to providing reliable, affordable, and results-oriented professional equipment. Since 2003, we have been providing superior equipment such as: microdermabrasion, microcurrent, oxygen infusion, nue skin scrubber + sonic wave and LED light therapy along with performance professional skin care.

Our parent company Lifeline Medical, Inc. has been in business since 1991. A strong commitment to customer service has contributed to our success. I, as owner and President, pledge to continue this tradition and to make Skin for Life the premier customer service oriented company in this industry.

Our dedicated employees understand the importance of customer focus and we treat all our customers with integrity and respect. We realize that our ability to grow as a company is directly related to customer satisfaction and that you make the final choice when selecting a vendor.

We look forward to the opportunity to support your business with quality equipment, supplies and skin care. Thank you for giving us a chance to earn your trust.



Tushar Shah

President and Founder of Skin for Life

# PURE O2 PORTABLE

## oxygen infusion

Important Safety Information .....	1
How Does Pure O2 Work? .....	2
Areas of Application / Treatments Required .....	3
Contraindications & Precautions .....	4
Machine Set up Instructions .....	5
PURE O2 Portable Machine Installation .....	6
Warranty Policy .....	7-8
Consent Form-Sample Only .....	9
Procedure Information Consent .....	10
L.E.D. Light Therapy Information .....	11
LED Light Therapy Consent Form-Sample Only .....	12
Professional Skin Care .....	13
Age Defying Skin Protocol .....	14-17
Simply Sensitive Skin Protocol .....	18-22
Oily & Acne Skin Protocol .....	23-26
Photo Aging Skin Protocol .....	27-30
Cost Per Treatment .....	31

# PURE O2 PORTABLE

## oxygen infusion



### IMPORTANT SAFETY INSTRUCTIONS

#### Operation Safety Instructions -

- Read all instructions before use to avoid injury
- To avoid serious injury or death, follow safety information in this document
- Keep this manual for reference
- Read all product safety labels
- Refer to local building codes for installation requirements
- Always turn off machine when not in use

#### Safety Information:

#### All Electrical Devices for Prevention of Electrical Shock Exposure -

- Keep machine unplugged when setting up machine to prevent accidental injury
- Make sure machine is on a clean and dry surface area to prevent any accidental injury
- Prohibit water from seeping into the machine housing
- Avoid any water and/or liquids around the vicinity of the machine when in use or not in use to prevent any accidental injury
- Check the cord and plug for damage before use
- Pay attention to safety labels on machine before plugging the cord into an outlet
- Switch on the equipment only after plugging into an outlet. Always turn off the machine before unplugging the cord from the outlet
- Allow sufficient space for ventilation to prevent overheating
- Avoid placing the cord near any hot objects

# PURE O2 PORTABLE OXYGEN INFUSION

Skin for Life's PURE O2 Portable oxygen infusion is a unique delivery system using advanced product technology. Our professional treatment skin care products are the main focus for skin change. We use chemistry to promote an assimilation of a pure oxygen environment. This assists skin cells to thrive and proliferate for a healthier life.

We use 20.9% oxygen (the same as we breathe) which is disbursed as consistent air flow from Skin for Life's Airbrush. The Oxygen Infusion activator phase 2 interacts with the natural enzymes within the stratum corneum. Enzymes create energy to breakdown the Oxygen Infusion phase 1 and Oxygen Infusion phase 2 into pure oxygen. Oxygen Infusion activator phase 2 is sprayed from the Airbrush for proper disbursement after the Oxygen Infusion Phase 1 application.

Our Oxygen Infusion phase 1 and Oxygen Infusion phase 2 are formulated for all skin types, conditions and colors.

## How Does Oxygen Infusion Work?

PURE O2 Infusion System has an output oxygen pump to dispense 20.9% oxygen onto the skin. On a cleansed and toned skin apply 7.5 ml/1.5 teaspoons from one of our four Oxygen Infusion phase 1 serums (A, C, E complex, antibacterial, brightening or sensitive). Use a facial mask brush to evenly apply a medium-thick application to face, neck and décolleté. Once application is completed pour 5 ml/1 teaspoon of Oxygen Infusion activator phase 2 into the airbrush cup, tightly close the lid.

Spray Oxygen Infusion activator phase 2 onto the skin in circular movements and activate the product by using the other hand. Massage the skin simultaneously while spraying the skin with Oxygen Infusion phase 2. The process is approximately 5-10 minutes or 3-4 passes to the face, neck and décolleté. Once passes have been completed massage the skin for 1-2 minutes. Remove with moistened room temperature towels and continue with skin treatment steps.

# PURE O2 PORTABLE OXYGEN INFUSION

## How Many Treatments Are Needed?

Oxygen Infusion may be incorporated with other modalities such as microdermabrasion, microcurrent, LED light therapy, skin scrubber + sonic wave or dermaplaning on a bi-monthly or monthly basis. Oxygen Infusion may also be used as a primary skin treatment.

When using oxygen infusion as a primary skin treatment schedule clients/patients twice a week for three weeks. This type of skin treatment plan allows clients/patients to reach their skin goals quickly for sensitive, pigmentation, oily/troubled or mature/aging skin. Clients may be seen every two weeks to maintain and sustain their skin goals.

## Who Benefits?

All skin types, conditions and colors may benefit from Oxygen Infusion Skin Treatment after a thorough client consultation from a skin care professional.

Our formulations are performance driven, providing healthy long-lasting skin results for a healthy youthful appearance.

## Areas of Application

- Face, Neck and Décolleté
- Shoulders and Back
- Hands and Arms
- Elbows, Knees & Feet

## Client Consultation

**INTAKE FORM** (sample included) - Skin for Life recommends a complete intake before performing any skin spa service.

Medical History • Medical Conditions • Allergies • Stress • Cosmetic Procedures •

Current Skin Care • Sensitivities • Topical Creams (Retin A, Differin, Tazarac, Renova, ... ) •

Medications - always ask your client/patient if they are taking any new medication(s).

# PURE O2 PORTABLE OXYGEN INFUSION

## Contraindications

- Accutane Users (Should be a minimum of 8 months) When in doubt obtain a physician release from your client.
- Pregnancy
- Breast Feeding
- Rosacea - Advanced Stages III
- Sunburn
- Cancer/Skin Cancer/Open Lesions
- Do Not Use in combination with TCA, Jessner or Salicylic Acid Cosmetic Acids

## Precautions

- Use caution and review cosmetic acid pH before performing Oxygen Infusion (Glycolic, Mandelic, Azelaic, Lactic, Citric) skin must be free from all sensitivity before applying Skin for Life's Oxygen Infusion
- Retinoid (Retin A, Tazorac, Differin or any vitamin A derivative topical product). Have client/patient discontinue for 3-5 days prior to performing an Oxygen Infusion Skin Treatment
- Skin Sensitivity
- Contacts - Please have clients/patients remove their contacts before procedure
- Under the care of a physician - general for all procedures administered



# PURE O2 PORTABLE OXYGEN INFUSION

## Set Up Instructions

**Check Package** for items listed below that come with your purchase:

☐ (1) Airbrush Gun with Hose	☐ (1) 1 oz. Bottle of Cleaner with Instructions
☐ (1) Power Cord	
<b>Oxygen Infusion Skin Care</b>	
☐ (1) PSSC-14 Oxygen Infusion A, C, E complex phase 1	☐ (1) PSSC-16 Oxygen Infusion brightening phase 1
☐ (1) PSSC-18 Oxygen Infusion activator phase 2	
<b>OPTIONAL - LED light therapy</b>	
☐ (1) RED LED light hand piece	☐ (1) BLUE LED light hand piece
<b>Additional Items</b>	
☐ (1) Operational Manual	☐ (50) Oxygen Infusion Rack Cards
☐ Optional (50) LED light Rack Cards	☐ (1) Price List & Order Form

# PURE O2 PORTABLE OXYGEN INFUSION

## Machine Installation



### Back View

#### Cord Placement

- Power Input is in the back of the machine
- Insert adapter in the back at the base of the machine
- Cord - DC power adapter has a 2 prong plug.

### On/Off Switch

#### Two Buttons

- LED light on/off switch only powers the LED light hand pieces
- O2 on/off switch only powers airbrush for dispersing oxygen

### LED light Optional

#### Hand Pieces

- The white cord from the LED light hand pieces have a 5 pin "Din Plug." View the LED socket to guide you for perfect alignment before inserting the plug into the LED socket



End of black hose. Insert into the front tube fitting for O2 Gun

Thread silver adapter to base of oxygen gun

### Airbrush

#### Attachment

- Thread silver hose adapter to the base of the oxygen gun
- Insert the black hose firmly into the front of the machine into the black fitting.
- When removing black hose use index finger and thumb to press down on the ring of black fitting and pull hose to remove

# PURE O2 PORTABLE OXYGEN INFUSION

## Warranty Policy

### Warranty Agreement –

- Skin for Life warrants this product to be free of defects in material and workmanship for a period of one year from the date of original purchase. **(Initial User Only – non transferable)**

### Responsibility of Original Purchaser –

- Retain Sales Receipt
- Keep Packaging (Box and Packing Material) for at least 2 years
- If machine requires off site service, contact Skin for Life authorized warranty service and technical center (866) 312.SKIN (7546)
- If equipment is brought back to Skin for Life under warranty and it is determined that lack of maintenance is the reason why equipment is compromised, customer will incur freight and repair costs

### Training Support –

- Equipment Training is mandatory to avoid improper use of equipment and to avoid compromising your warranty
- Call Skin for Life at 866-312-7546 to schedule your initial training which may take place on-site or by web conference

### Note\*

Maintenance is enforced along with proper training so you may maintain an excellent working machine . Without proper maintenance and training you could possibly **VOID** your warranty.

### Airbrush Accessory Warranty –

All Skin for Life accessories have a **90 day warranty**. If within 90 days there is a concern with the airbrush accessory, we will replace under the warranty agreement. However, compliance of the following steps must be followed:

1. Call Skin for Life (866) 312.7546. Customer service will give a RMA#. Write the number on the return package.
2. Mail defective airbrush through USPS (postal service)
3. Skin for Life will evaluate. If the airbrush is defective we will replace and absorb the cost of shipping for the inconvenience.

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

# PURE O2 PORTABLE OXYGEN INFUSION

## Warranty Guidelines

### This Warranty Does Not Cover -

- Merchandise sold as demo units, reconditioned, floor models and/or display models
- Merchandise used as “Rental” equipment
- Merchandise that has become inoperative because of misuse, improper and/or unauthorized repair or alterations or failure to operate product in accordance with the instructions provided in the “General Operating Manual” and reiterated in the “Mandatory” training received on-site or by webinar
- Repair and transportation costs if equipment is determined not to be defective.
- Any incidental, indirect, or consequential loss, damage or expense that may result from any defect, failure or malfunction of the products

### Skin for Life Return Policy -

- All returns and/or credits are subject to approval by Skin for Life and **MUST** be made within **30 days of the original sale**
- If merchandise is returned after 30 days or without proper authorization then it is Skin for Life’s prerogative to **charge a 25% restocking fee**
- Customers incur all “Shipping Costs” for the return of equipment, supplies and/or skin care to Skin for Life
- Everything must be returned in original **“New”** condition
- Skin for Life only accepts unopened and untampered skin care product as a return. A **25% restocking fee may be applied**
- All accessories must be returned with equipment
- Once received, Skin for Life will evaluate the equipment and accessories. Once confirmed, credit will be issued to customer. **Credit Memo** will be emailed to customer for confirmation of the refunded amount

# Consent Form - Sample ONLY

First Name:		Last Name:	
Date of Birth:    /    /			
Address:			
City:		State:	Zip:
Preferred Contact Number:			
Emergency Contact:		Phone:	
Referred By:			
1. What is the reason for your visit today? _____			
2. Are you under the care of a physician including dermatologist?			
3. Are you pregnant? _____ Are you planning to become pregnant? _____			
4. Do you smoke? _____			
5. Please circle the one that best describes your daily stress level?    High    Medium    Low			
6. Do you have metal in your body? dental implants, pins, plates, stents, etc. Be specific -			
7. Do you have allergies?			
8. Please list ALL Medications - _____ _____			
9. Do you use any exfoliants? Retin-A, Tretinoic acid, glycolic acid, scrubs, vitamin A derivatives Please specify _____			
10. Do you have or have you had the following: Please circle all that apply:			
Heart Problems	High Blood Pressure	Heart Stent	Mitral Valve Prolapse
Pace Maker	Epilepsy	Fibromyalgia	Lupus
Hepatitis B or C	Immune Disorder	Kidney Disorder	Diabetes
Rosacea	Acne	Skin Cancer	Herpes Zoster (shingles)
Cosmetic Surgery	Botox	Cosmetic Fillers	Asthma
Keloid	Cancer	Sinus Concerns	Fever Blisters

I understand the information I have given is to aid the skin therapist and is not a substitute for medical care. I understand these questions and I have answered them honestly.

Client Signature \_\_\_\_\_

Date \_\_\_\_\_

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

# Procedure Information Consent - Sample ONLY

## Instructions -

- **Do not** wear contact lenses during treatment sessions - they may be placed back in after treatment
- **Do not** use glycolic, AHA's, or any retinol products 48 hours before or after treatment
- You must not be currently taking Accutane. No Accutane use for at least 8 months to 1 year
- No suntan beds for 2 weeks post treatment (recommend self tanners for a healthy lifestyle change)
- Apply SOL defense SPF 30 physical sunscreen after treatment and any time skin during sun exposure

## Initial Required -

\_\_\_ I have been thoroughly informed by professional therapist \_\_\_\_\_ of any complications to recommended procedure \_\_\_\_\_ and I understand there are no guarantees to procedures given by \_\_\_\_\_.

\_\_\_ I am committed to following post treatment care given to me by professional therapist \_\_\_\_\_ to ensure the best possible result.

\_\_\_ I have given all medical information to the best of my knowledge to \_\_\_\_\_ to prevent any contraindication to procedures.

Yes or No - I have had history of cold sores, fever blisters and or shingles.

Yes or No - I am using (Retin-A, Tretinoic acid, Differin, glycolic acid, Accutane (isotretinoin) or other vitamin A derivative topical creams and/or ointments.

\_\_\_ I have not used any performance agents within the last 5-7 days, knowing that this is a contra indicator for having procedures performed.


\_\_\_ I am committed to avoiding the sun and/or tanning facilities for the recommended time suggested by professional therapist \_\_\_\_\_.

Client Signature \_\_\_\_\_ Date \_\_\_\_\_

Professional Signature \_\_\_\_\_ Date \_\_\_\_\_

# LED Light Therapy Information


LED technology was originally developed to enhance the growth of plant tissue in space by NASA's Marshall Space Flight Center and Quantum Devices, Inc. of Barneveld, Wisconsin. LED's have a similar physiological effect on human cells as they do on plant cells. It is a natural photo - biochemical reaction that is similar to photosynthesis in plants, which triggers the body to convert light energy into cell energy.

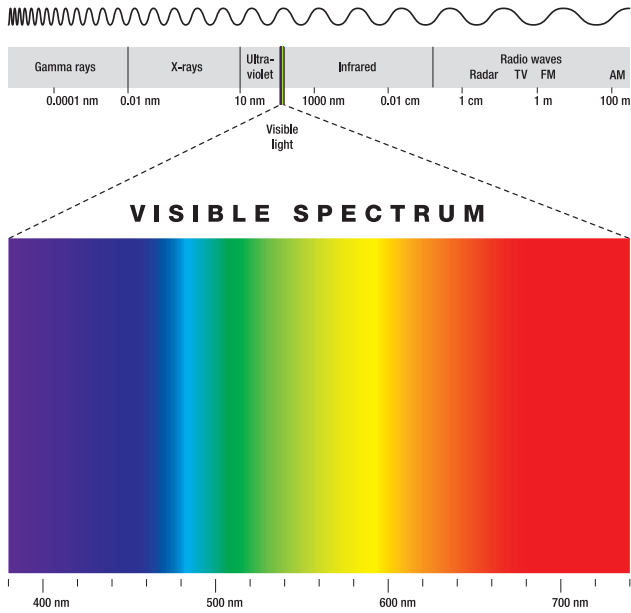

**RED LIGHT  
(680 nm)  
Age Defying**

**BLUE LIGHT  
(460 nm)  
Oily - Acne Skin**

**INFRA-RED LIGHT  
(800 nm)  
Skin Rebuilding**

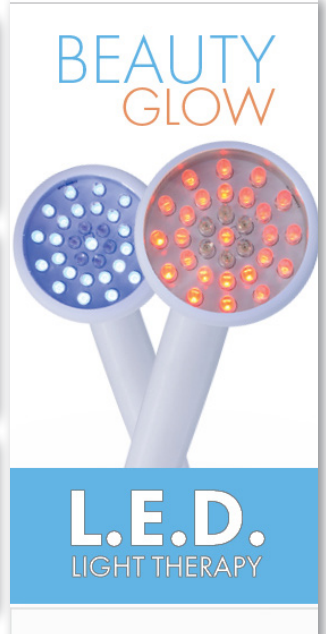

This remarkable technology enhances the ATP (adenosine triphosphate) within the mitochondria of the cells which excites the first process of cell division. This non thermal response relies on the visible light spectrum to interact with cells for a growth response.

**RED LED Light** is designed to trigger the first initial ATP (adenosine triphosphate) response.

RED LED light should always be the first light used on all skin types, conditions and colors.

**Skin Indications-** sun damaged skin, mature, pigmented lesions, oily, troubled skin, capillary distention and temporary/chronic sensitive skins.

**How To Use - RED** LED light hand piece - use 90 seconds per area with overlapping 30%-50%. Approximately 4-5 passes to complete treatment.

**BLUE LED Light** is suggested for oily and troubled skin conditions. The depth of this light is important to minimize the activity of bacteria within the sebaceous gland that is associated with blemished skins.

**Skin Indications -** oily, troubled skin, blemished skin and seborrhea.

**How To Use -** BLUE LED light hand piece - use for 90 seconds per area with overlapping 30% - 50%. Approximately 4-5 passes to complete treatment.

# LED Light Therapy

## Procedure Information Consent

The purpose of the LED light therapy study from NASA was to discover the properties of wound healing. Increasing cellular growth encourages cell division which provides skin with a youthful healthy appearance. LED light therapy also assists with minimizing overactive oily and/or troubled skin conditions.

### Ideal Results -

Schedule twice a week for three weeks which equals (6) skin treatments.

### Maintenance Treatments -

Depends on individual goals. Once a week, twice a month or once a month is recommended to sustain cumulative results.

### LED Light Therapy Contraindication -

- Epilepsy
- Thyroid Conditions
- Pregnancy
- **Do Not** apply to an open wound

### Precautions to Consider -

- Sensitivity to Migraines or Headaches
- **Do Not** use heavy cream and/or ointments
- Topical or Systemic Steroids

LED Light Therapy treatments have been explained to me and any questions I have regarding LED light therapy treatments have been answered.

I understand there are certain contraindications that preclude someone from receiving LED light therapy treatments. These include, but are not limited to epilepsy, pregnancy, thyroid conditions, medication causing light sensitivity and open wounds.

I also understand there are other precautions that should be considered before receiving LED light treatments and may require a doctor's release and/or I assume any risk involved. These include but not limited to a person receiving cortisone and/or steroid injections, and a person prone to light sensitive migraines.

Although LED light therapy is a noninvasive treatment and the precautions and contraindications have been explained to me, I understand that certain risks or complications can occur from known and unknown causes and I freely assume these risks.

I acknowledge no guarantee has been given to me as to the condition of the complexion, skin pore size, fine lines and wrinkles as results will vary per client/patient. I am aware that follow up treatments may be necessary to achieve optimal results with gradual results occurring over time.

I release \_\_\_\_\_, staff and specific spa technician from any liability associated with this procedure. I certify that I am at least 18 years old or I have parental consent as indicated as co - signature below.

Client Signature \_\_\_\_\_

Date \_\_\_\_\_

Co-Signature \_\_\_\_\_

Date \_\_\_\_\_

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.



# PURE O2 PORTABLE OXYGEN INFUSION

Professional Skin Care + LSS™ delivery



When Skin for Life started manufacturing machines, we knew as a company that skin care should be a rebuilding process from progressive treatments like microdermabrasion. We started working with cosmetic experts to formulate our immunity professional skin care. Our Skin for Life pure science skin care repairs, recovers and restores all skin types, conditions and colors without inflammation and long-term irritation.

We wanted building blocks for long term healthy skin results. Our product line removes the stress, so all skin may proceed with a normal, healthy function. Skin for Life trademarked LSS™ delivery “Life Sustaining System” enriches our formulas by providing key amino acids and peptides that are immunity driven for your skin’s protection and well being. Our ingredients are the innovative performance that makes Skin for Life pure science perfect for every skin condition, type and color.

It is our pleasure to introduce you to Skin for Life pure science. You may add our signature treatments to your menu and introduce healthy results to your clients/patients.

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

# AGE DEFYING PROTOCOL

Age Defying Skin Treatment from Skin for Life is designed for mature and aging skin. The skin may appear to have wrinkles, fine lines, loss of elasticity, thin skin or uneven texture.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



Apply 5 ml / 1 teaspoon on dry skin. Massage for 1-2 minutes. Wet your hands and massage one more minute.

Remove carefully with moistened room temperature towels or compressed sponges.

You may remove eye makeup during your cleansing step. Remove the eye makeup gently with moistened cotton rounds.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.

## EXFOLIATION RECOMMENDATIONS

- Enzyme Exfoliant papaya complex + LSS™ delivery
- Skin Scrub + marine algae extracts
- Nue skin scrubber + sonic wave
- Microdermabrasion
- Dermaplaning
- Pro Radiance Peel active brighteners + LSS™ delivery
- Professional Peel salicylic, lactic, citric + LSS™ delivery

*Please see each product's protocol for additional information. Visit [skinforlife.com](http://skinforlife.com) or call (866) 312.7546.*

# AGE DEFYING PROTOCOL

Age Defying Skin Treatment from Skin for Life is designed for mature and aging skin. The skin may appear to have wrinkles, fine lines, loss of elasticity, thin skin or uneven texture.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.



Apply 7.5 ml/ 1 1/2 teaspoon on dry skin.

Apply an even, medium- thick application to face, neck and décolleté with a facial mask brush of the Oxygen Infusion A, C, E complex. Avoid direct eye area.

Use effleurage and lifting massage movements. Massage for 1-2 minutes. This will excite the skin's receptivity. Do Not Remove.



Use 5 ml/ 1 teaspoon. Pour into an airbrush applicator cup.

Start with airbrush movements. Use small to medium circular movements over the Oxygen Infusion Gel phase 1.

Make at least 3 passes while manipulating the Oxygen Infusion Gel phase 1 with finger tips. This will ensure a thorough application.

Once completed, massage for another 1-2 minutes using effleurage, petrissage or lifting massage movements.

Remove with moistened room temperature towels or compressed sponges.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.

# AGE DEFYING PROTOCOL

Age Defying Skin Treatment from Skin for Life is designed for mature and aging skin. The skin may appear to have wrinkles, fine lines, loss of elasticity, thin skin or uneven texture.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



On dry skin apply RED L.E.D. Light hand piece for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another skin area. Continue with RED L.E.D. Light placement until application is complete.

If “Acne or Oily Skin” is present continue with the BLUE L.E.D. Light hand piece and apply the same above steps over the skin areas of concern until application time is completed.

Note\*

Avoid direct contact with eye area and thyroid area.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté. May apply around the eye area for a hydrating eye serum.

Massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after the Skin for Life Performance Serum Application. Apply to face, neck and décolleté. Blend well.

May use as an intensive antioxidant cream for photo-damaged skin.

May use lifting massage movements to firm and tighten the skin.

## MASK RECOMMENDATIONS

- CryoAlgae Mask + CryoAlgae Booster
- Charcoal - Anti Pollution Mask
- Revitalizing Mint Mask purifying camphor + LSS™ delivery

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

# AGE DEFYING PROTOCOL

Age Defying Skin Treatment from Skin for Life is designed for mature and aging skin. The skin may appear to have wrinkles, fine lines, loss of elasticity, thin skin or uneven texture.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté. May apply around the eye area for a hydrating eye serum.

Massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after the Skin for Life Performance Serum Application. Apply to face, neck and décolleté. Blend well.

May use as an intensive antioxidant cream for photo-damaged skin.

May use lifting massage movements to firm and tighten the skin.



Apply 2.5 ml/ 1/2 teaspoon as the final treatment step after a Skin for Life moisturizer application.

Apply to face, neck and décolleté. Blend well.

May use on face and body for sunscreen protection.

# SIMPLY SENSITIVE PROTOCOL

Simply Sensitive Skin Treatment from Skin for Life is designed for temporary or chronic sensitive skin conditions. The skin may appear to have redness, dryness, chapped skin or capillary distention (telangiectasia).

You may perform a series of 3-6 treatments, depending on the sensitivity. Recommend clients schedule two weeks apart for skin repair, recovery and optimal results.



Apply 5 ml/1 teaspoon on dry skin. Massage for 1-2 minutes.

Moisten hands and massage for another minute.

Remove with moistened room temperature towels or compressed sponges.

Dry skin thoroughly.

Note\*

May pour into a glass bowl and mix with a facial mask brush. Apply and begin a light massage. Complete with above steps.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.

## EXFOLIATION RECOMMENDATIONS

- Exfoliant Enzyme papaya complex + LSS™ delivery
- Skin Scrub + marine algae extracts
- Nue skin scrubber + sonic wave
- Dermaplaning
- Nue Skin Organic Grains Microdermabrasion
- Nue Skin Microdermabrasion (crystals) – adjust vacuum and crystals according to sensitivity

*Please see each product's protocol for additional information. Visit [skinforlife.com](http://skinforlife.com) or call (866) 312.7546*

# SIMPLY SENSITIVE PROTOCOL

Simply Sensitive Skin Treatment from Skin for Life is designed for temporary or chronic sensitive skin conditions. The skin may appear to have redness, dryness, chapped skin or capillary distention (telangiectasia).

You may perform a series of 3-6 treatments, depending on the sensitivity. Recommend clients schedule two weeks apart for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.



Apply 7.5 ml/ 1 1/2 teaspoon on dry skin.

Apply an even, medium- thick application to face, neck and décolleté with a facial mask brush.

- Use light effleurage massage movements. Massage for 1-2 minutes. This will excite the skin's receptivity. Do Not Remove.



- Pour 5 ml/ 1 teaspoon of Oxygen Infusion activator phase 2 into an airbrush applicator cup.
- Start with airbrush movements. Use small to medium circular movements over the Oxygen Infusion sensitive phase 1.
- Make at least 3 passes while manipulating the Oxygen Infusion sensitive with finger tips. This will ensure a thorough application.
- Once completed, massage for another 1-2 minutes using light effleurage massage movements.
- Remove with moistened room temperature towels or compressed sponges.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.

# SIMPLY SENSITIVE PROTOCOL

Simply Sensitive Skin Treatment from Skin for Life is designed for temporary or chronic sensitive skin conditions. The skin may appear to have redness, dryness, chapped skin or capillary distention (telangiectasia).

You may perform a series of 3-6 treatments, depending on the sensitivity. Recommend clients schedule two weeks apart for skin repair, recovery and optimal results.



On dry skin apply RED L.E.D. Light hand piece for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another skin area. Continue with RED L.E.D. Light placement until application is complete.

If “Acne or Oily Skin” is present continue with the BLUE L.E.D. Light hand piece and apply the same above steps over the skin areas of concern until application time is completed.

Note\*

Avoid direct contact with eye area and thyroid area.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté.

May apply around the eye area for a hydrating eye serum.

Perform a light effluerage massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after Vita C Peptide Serum.

Apply to face, neck and décolleté. Blend well. May use as a healing cream for sensitive skins.

Perform light effluerage massage movements until moisturizer is absorbed.



Apply 2.5 ml/ 1/2 teaspoon after Vital Recovery application.

Apply to face, neck and décolleté. Blend well.

May use as an intensive neck cream for sagging skin.

May use as a light massage medium with massage techniques.



# SIMPLY SENSITIVE PROTOCOL

Simply Sensitive Skin Treatment from Skin for Life is designed for temporary or chronic sensitive skin conditions. The skin may appear to have redness, dryness, chapped skin or capillary distention (telangiectasia).

You may perform a series of 3-6 treatments, depending on the sensitivity. Recommend clients schedule two weeks apart for skin repair, recovery and optimal results.

## MASK RECOMMENDATIONS

- Revitalizing Mint Mask purifying camphor + LSS™ delivery
- CryoAlgae Mask + CryoAlgae Booster

Please see each product's protocol for additional information. Visit [skinforlife.com](http://skinforlife.com) or call (866) 312.7546



Apply 2.5 ml / 1/2 teaspoon on dry skin. It can be used as a spray or saturate a cotton round.

Dry the skin thoroughly.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté. May apply around the eye area for a hydrating eye serum.

Massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after Vita C Peptide Serum.

Apply to face, neck and décolleté. Blend well. May use as a healing cream for sensitive skins.

Perform light effluerage massage movements until moisturizer is absorbed.

# SIMPLY SENSITIVE PROTOCOL

Simply Sensitive Skin Treatment from Skin for Life is designed for temporary or chronic sensitive skin conditions. The skin may appear to have redness, dryness, chapped skin or capillary distention (telangiectasia).

You may perform a series of 3-6 treatments, depending on the sensitivity. Recommend clients schedule two weeks apart for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon after Vital Recovery application.

Apply to face, neck and décolleté. Blend well.

May use as an intensive neck cream for sagging skin.

May use as a light massage medium with massage techniques.



Apply 2.5 ml/ 1/2 teaspoon as the final treatment step after a Skin for Life moisturizer application.

Apply to face, neck and décolleté. Blend well.

Choose from one of our tints natural, medium or dark.

# OILY & ACNE SKIN PROTOCOL

Oily Skin Treatment from Skin for Life is designed for oily, thick and troubled skin. The skin may appear to have excessive oil, open/closed comedones, milia cyst, Grade I-II acne or thick stratum corneum layer.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



Apply 5 ml/1 teaspoon on dry skin. Massage for 1-2 minutes. Moisten hands and massage for another minute.

Remove carefully with moistened room temperature towels or compressed sponges.

Avoid direct eye contact.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May saturate a cotton round or pad. Apply to face, neck and décolleté as an even application. Dry the skin thoroughly.

Note\*

If skin is “oily-sensitive”, may use Instant Soothing Toner natural actives as the treatment toner.

## EXFOLIATION RECOMMENDATIONS

- Enzyme Exfoliant papaya complex + LSS™ delivery
- Skin Scrub + marine algae extracts
- Nue skin scrubber + sonic wave
- Microdermabrasion - use for non-active acne only
- Pro Radiance Peel active brighteners + LSS™ delivery
- Professional Peel salicylic, lactic, citric

*Please see each product's protocol for additional information. Visit [skinforlife.com](http://skinforlife.com) or call (866) 312.7546 for details.*

# OILY & ACNE SKIN PROTOCOL

Oily Skin Treatment from Skin for Life is designed for oily, thick and troubled skin. The skin may appear to have excessive oil, open/closed comedones, milia cyst, Grade I-II acne or thick stratum corneum layer.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or every two weeks for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.



Apply 7.5 ml/ 1 1/2 teaspoon on dry skin.

Apply an even, medium- thick application to face, neck and décolleté with a facial mask brush. Avoid eye area.

Use petrissage or tapotement massage movements. Massage for 1-2 minutes. This will excite the skin's receptivity.

Do Not Remove.



Use 5 ml/ 1 teaspoon. Pour into an airbrush applicator cup.

Start with airbrush movements. Use small to medium circular movements over the Oxygen Infusion Gel phase 1.

Make at least 3 passes while manipulating the Oxygen Infusion Gel phase 1 with finger tips. This will ensure a thorough application.

Once completed, massage for another 1-2 minutes using effleurage, petrissage or lifting massage movements.

Remove with moistened room temperature towel(s) or compressed sponges.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application.

Dry skin thoroughly.

# OILY & ACNE SKIN PROTOCOL

Oily Skin Treatment from Skin for Life is designed for oily, thick and troubled skin. The skin may appear to have excessive oil, open/closed comedones, milia cyst, Grade I-II acne or thick stratum corneum layer.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.



On dry skin apply RED L.E.D. Light hand piece for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another skin area. Continue with RED L.E.D. Light placement until application is complete.

If “Acne or Oily Skin” is present continue with the BLUE L.E.D. Light hand piece and apply the same above steps over the skin areas of concern until application time is completed.

Note\*

Avoid direct contact with eye area and thyroid area.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté. May apply around the eye area for a hydrating eye serum. Massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after the Skin for Life Performance Serum application.

Apply to face, neck and décolleté. Blend well. May be used in target areas of concern.

Perform a light massage until Antibac Essentials is absorbed.



Apply 5 ml/ 1 teaspoon after Antibac Essential application.

Apply to face, neck and décolleté. Blend well. May use as a healing cream for oily-sensitive skins.

Perform a light massage until Vital Recovery is absorbed.

May use as an extra eye cream for the delicate eye area.

©2019 all rights reserved  
Skin for Life a division of Lifeline Medical, Inc.

# OILY & ACNE SKIN PROTOCOL

Oily Skin Treatment from Skin for Life is designed for oily, thick and troubled skin. The skin may appear to have excessive oil, open/closed comedones, milia cyst, Grade I-II acne or thick stratum corneum layer.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.

## MASK RECOMMENDATIONS

- Revitalizing Mint Mask purifying camphor + LSS™ delivery
- Charcoal - Anti Pollution Mask
- CryoAlgae Mask + CryoAlgae Booster



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application. Dry skin thoroughly.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté.

May apply around the eye area for a hydrating eye serum.

Massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon to face, neck and décolleté. Blend well.

May use around the delicate eye area for extra hydration.



Apply 2.5 ml/ 1/2 teaspoon as the final treatment step after Vital Recovery Concentrate application.

Apply to face, neck and décolleté. Blend well.

# PHOTO AGING PROTOCOL

Photo-Aging Skin Treatment from Skin for Life is designed for pigmented skin conditions. The skin may appear to have melasma, hyperpigmentation, sun lentigines or other related sun pigmented conditions.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.



Apply 5 ml/1 teaspoon on dry skin. Massage for 1-2 minutes.

This product is safe to use as an eye makeup remover.

Moisten hands and massage for another minute.

Remove with moistened room temperature towels, compressed sponges or esthetic gauze.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application. Dry skin thoroughly.

## EXFOLIATION RECOMMENDATIONS

- Pro Radiance active brighteners + LSS™ delivery
- Professional Peel salicylic, lactic, citric
- Skin Scrub + marine algae extracts
- Enzyme Exfoliant papaya complex + LSS™ delivery
- Dermaplaning
- Microdermabrasion 1 pass Crystals & 1 pass Organic Walnut Grains
- Nue skin scrubber + sonic wave

*Please see each product's protocol for additional information. Visit [skinforlife.com](http://skinforlife.com) or call (866) 312.7546 for details.*

# PHOTO AGING PROTOCOL

Photo-Aging Skin Treatment from Skin for Life is designed for pigmented skin conditions. The skin may appear to have melasma, hyperpigmentation, sun lentigines or other related sun pigmented conditions.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application. Dry skin thoroughly.



Apply 15 ml/ 1 tablespoon on dry skin.

Apply an even application to face, neck and décolleté with esthetic gauze. Avoid eye area.

DO NOT REMOVE.

Dry skin thoroughly.



Apply 7.5 ml/ 1 1/2 teaspoon on dry skin after an exfoliation application.

Apply an even medium- thick application to face, neck and décolleté with a facial mask brush. Avoid eye area.

Use effleurage and lifting massage movements. Massage for 1-2 minutes. This will excite the skin's receptivity. Do Not Remove.



Use 5 ml/ 1 teaspoon. Pour into an airbrush applicator cup.

- Start with airbrush movements. Use small to medium circular movements over the Oxygen Infusion Gel phase 1.
- Make at least 3 passes while manipulating the Oxygen Infusion Gel phase 1 with finger tips. This will ensure a thorough application.
- Once completed, massage for another 1-2 minutes using effleurage, petrissage or lifting massage movements.
- Remove with moistened room temperature towel(s) or compressed sponges.



# PHOTO AGING PROTOCOL

Photo-Aging Skin Treatment from Skin for Life is designed for pigmented skin conditions. The skin may appear to have melasma, hyperpigmentation, sun lentigines or other related sun pigmented conditions.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application. Dry skin thoroughly.



On a dry skin apply RED L.E.D. Light hand piece for 90 seconds and transition with a 30% - 50% overlap by placing the hand piece to another skin area. Continue with RED L.E.D. Light placement until application is complete.

BLUE LED light may be used for oily, acne or blemished skin. Apply for additional 10 minutes for areas of concern.

Note\* Avoid direct contact with eye area and thyroid area.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté.

May apply around the eye area for a hydrating eye serum.

Perform effleurage lifting massage until serum is absorbed.



Apply 5 ml/ 1 teaspoon after Vita C Peptide Serum application. Apply to face, neck and décolleté. Blend well.

Perform effleurage lifting massage or lymphatic drainage for skin performance.

May use as an intensive neck and décolleté cream for skin firming.

## MASK RECOMMENDATIONS

- Revitalizing Mint Mask purifying camphor + LSS™ delivery
- Charcoal - Anti Pollution Mask
- CyroAlgae Mask + CryoAlgae Booster

Please see each product's protocol for additional information. [skinforlife.com](http://skinforlife.com)

©2019 all rights reserved

©2019 Skin for Life division of Lifeline Medical, Inc.

Skin for Life a division of Lifeline Medical, Inc.

# PHOTO AGING PROTOCOL

Photo-Aging Skin Treatment from Skin for Life is designed for pigmented skin conditions. The skin may appear to have melasma, hyperpigmentation, sun lentigines or other related sun pigmented conditions.

You may perform a series of 6 treatments. Recommend clients schedule twice a week for 3 weeks or two weeks apart for skin repair, recovery and optimal results.



Apply 2.5 ml/ 1/2 teaspoon on dry skin. May use as a spray or saturate a cotton round or pad.

Apply to face, neck and décolleté as an even application. Remove mask residue thoroughly.

Dry skin thoroughly.



Apply 1.25 ml/ 1/4 teaspoon on dry skin. Apply to face, neck and décolleté.

May apply around the eye area for a hydrating eye serum.

Perform effleurage lifting massage until serum is absorbed.



Apply 2.5 ml/ 1/2 teaspoon after Vita C Peptide Serum application.

Apply to face, neck and décolleté. Blend well.

May use as an intensive neck and décolleté cream for loose, sagging skin.

Apply around the delicate eye area as a second skin protection against dehydration.



Apply 2.5 ml/ 1/2 teaspoon as the final treatment step after a Skin for Life moisturizer application.

Apply to face, neck and décolleté. Blend well.

# PURE O2 Portable Oxygen Infusion

## Cost Per Treatment

Product Item #	Product Cost	Cost Per Treatment
Oxygen Infusion A,C,E complex phase 1	\$33.00	\$2.06 per treatment
Oxygen Infusion antibacterial phase 1	\$33.00	\$2.06 per treatment
Oxygen Infusion brightening phase 1	\$33.00	\$2.06 per treatment
Oxygen Infusion sensitive phase 1	\$33.00	\$2.06 per treatment
Oxygen Infusion activator phase 2	\$45.00	\$1.25 per treatment
<b>Total Cost Per Each Treatment (1) phase 1 + phase 2</b>		<b>\$3.31 per treatment</b>

The **number one spa treatment** is the number one most profit driven treatment for licensed professionals. You may build your treatments with other advanced performance therapies such as:

- Nue Skin Microdermabrasion
- LED Light Therapy
- Nue Skin Scrubber + Sonic Wave Equipment
- Microcurrent
- Dermaplaning
- Skin for Life Professional Peel System
- Skin Scrub + marine algae extracts
- Enzyme Exfoliant papaya complex + LSS™ delivery

Advanced skin treatments are the building blocks for successful licensed skin care professionals. Incorporating innovative treatment techniques bring instant, long-term and healthy skin results.

Skin for Life brings innovation, quality and result-oriented products to licensed professionals.

The logo features a stylized grey eye icon on the left, followed by the text "skin for life" in a sans-serif font. "skin" and "for" are in grey, while "life" is in green. Below this, the word "PROFESSIONAL" is written in all caps, spaced out, in a smaller grey font.

skin for life  
P R O F E S S I O N A L

15241 TELCOM DR.  
BROOKSVILLE, FL 34604



No Animal Testing